EN
BTL RESPIRATOR User's manual
FLAT-FIT Healthcare Respirator

ES
BTL RESPIRATOR Manual del usuario
FLAT-FIT Healthcare Respirator

BTL Industries Ltd.
161 Cleveland Way
Stevenage
Hertfordshire
SG1 6BU
United Kingdom

905-80MANDU01HREU08100

www.bodybybtl.com

EN BTL RESPIRATOR User's manual
FLAT-FIT Healthcare Respirator

BASIC CHARACTERISTICS AND DIRECTIONS FOR USE

Directions for proper application

- Slightly bent and pre-shape the nosepiece of the respirator.
- Separate the edges of the respirator to fully open it. Gently extend the elastic headbands one after another.
- While holding the headbands with your index fingers and thumbs, cup the respirator under your chin with nosepiece facing upwards and pull the headbands up over your head.
- Release the lower headband and position it at the base of your neck than position the remaining headband around of your head above the ears.
- Bend the nosepiece to the shape of your nose by firmly pressing down on both sides with your fingers.
- Continue to adjust the respirator and secure the edges until you feel you have achieved a good facial fit. Now, perform a check for proper sealing.

Ensure that the package is not damaged or contaminated in any way before use. Before fitting your respirator, you must determine there are no holes, cracks or visible damage around the seams and welds of the respirator.

CHECK OF PROPER SEALING

Place both hands as a cup over the respirator. Inhale and exhale sharply. Be careful not to disturb the position of the respirator. If air leaks around your nose, re-adjust the nosepiece as described in Step 5. If air leaks around respirator edges, adjust panels and position of straps and make certain respirator edges fit snugly against the face. If you cannot achieve a proper seal, do not enter the contaminated area. Seek and ask your supervisor.

DIRECTIONS FOR PROPER REMOVAL

- Without touching the respirator facepiece slowly lift the bottom strap from around your neck up over your head.
- Lift off the top strap. Do not touch the respirator.
- Store or discard according to your facility's infection control policy.

Do not touch the surface of the product, it may be contaminated with hazardous particles. Treat the used product as infectious waste.

INTENDED USE

Filtering half mask to protect against potentially hazardous and noxious particles in the air.

CHARACTERISTICS OF USE

The product is approved and designed to provide respiratory protection by filtering potentially hazardous and noxious particles from the air during respiration, thereby reducing user exposure to these particles when used in accordance with all instructions and restrictions of use in the instructions for use and applicable safety precautions, and health regulations.

The product is placed on the market in accordance with the requirements of the Regulation (EU) 2016/425. The product is placed on the market in accordance with the requirements of the standard EN 149:2001+A1:2009 in FFP2 NR class.

In case of use for reduction of exposures to certain airborne particles (biological particles, dust, etc.), product can reduce, but cannot eliminate the risk of contracting infection, illness or disease.

SAFETY PRECAUTIONS

Read the User's Manual carefully and become familiar with all its safety requirements, operating procedures and maintenance instructions prior to use of the product.

- It is prohibited to use the product in any manner that is not in accordance with the User's Manual.
- If you suffer from asthma or other respiratory disease, consult your doctor for the use of respirators.
- Do not use in atmospheres containing less than 19.5% oxygen.
- Do not alter, abuse or misuse this respirator.
- If the respirator becomes damaged, leave the contaminated area immediately and repair or replace the respirator.
- Not for use in atmospheres immediately dangerous to life or health.
- Do not exceed maximum use concentrations established by regulatory standards.
- Failure to properly use and maintain this product could result in injury or death.
- Never substitute, modify, add, or omit parts. Use only exact replacement parts in the configuration as specified by the manufacturer.
- This product must not be used for more than 8 hours (working shift). Always discard and replace the respirator, when the breathing resistance becomes excessive or at the end of every working shift.
- The wearer of the respirator must be trained to use properly before starting work.

Do not use the respirator as protection against toxic particles, gases or vapors, solvents from spray-painting operations, or in an explosive atmosphere.

Do not use this respirator in tanks, enclosed places, or in any circumstances where gas is likely to be presented or in oxygen-deficient atmospheres.

This respirator should not be worn when conditions prevent a good face to facepiece seal. Such conditions include the growth of beards or mustaches and sideburns and the use of spectacles or any other devices containing temples or straps which pass between the respirator sealing edge and the wearer's face.

Leave the working area immediately if breathing becomes difficult or dizziness or any other signs of distress occur.

It is not allowed to use the respirator without a proper fit.

The instruction for use and warning is helpful for the correct use of the product and to prevent risks of death. It is peremptory to read it carefully and to follow it.

The respirator that has been used must not be used by another person.

SYMBOLS AND MARKING OF THE DEVICE

▲ = Warning, △ = Caution, ↓ = Temperature range of storage conditions. ↕ = Maximum relative humidity of storage conditions. ⊕ = The product is not reusable. Single shift use only. [I] = Follow instructions for use (User's manual) [M] = Manufacturer, [D] = Date of manufacture, [REF] = Catalogue number, [LOT] = Batch code, [QTY] = Quantity, [E] = Expiration date. End of shelf life. CE = CE mark.

MAINTENANCE AND STORAGE OF THE PRODUCT

Keep the respirator clean. Do not use disinfectants for cleaning and discard after use. The product is not intended for sterilization before use or for repeated use. This product is intended for a maximum of one working shift. The shelf life of the respirator is 5 years and the respirator should be placed in a container and stores in a clean cool and dry atmosphere to ensure that it is protected against dust, chemicals, moisture, excessive heat, and direct sunlight.

Do not use the product after expiration the date stated on the package.

MANUFACTURER

BTL Industries Ltd., 161 Cleveland Way, Stevenage, Hertfordshire, SG1 6BU United Kingdom. E-mail: sales@btlnet.com. Declaration of conformity is available online at: www.btlnet.com.





CONFORMITY ASSESSMENT AND NOTIFIED BODY

The conformity assessment of this product according to the procedures specified in Module B together with the procedure specified in Module C2 of Regulation (EU) 2016/425 was performed by a notified body:

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Date of last revision: 17th July 2020. ID: 905-80MANDU01HREU08100

© All rights reserved. No part of this manual may be reproduced, saved or transferred by any means including electronic, mechanical, and photographic without the prior written approval from BTL Industries Limited.

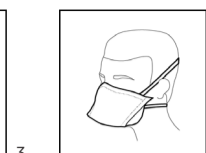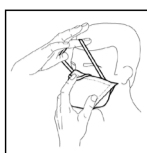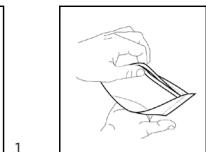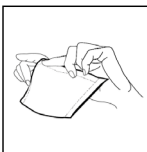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