



April 21, 2020

Instylla, Inc.  
Jennifer Greer  
Regulatory Affairs Manager  
204 2nd Avenue  
Waltham, Massachusetts 02451

Re: K200744  
Trade/Device Name: Instylla Microcatheter  
Regulation Number: 21 CFR 870.1210  
Regulation Name: Continuous Flush Catheter  
Regulatory Class: Class II  
Product Code: KRA  
Dated: March 20, 2020  
Received: March 23, 2020

Dear Ms. Greer:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. Although this letter refers to your product as a device, please be aware that some cleared products may instead be combination products. The 510(k) Premarket Notification Database located at <https://www.accessdata.fda.gov/scripts/cdrh/cfdocs/cfpmn/pmn.cfm> identifies combination product submissions. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part

801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803) for devices or postmarketing safety reporting (21 CFR 4, Subpart B) for combination products (see <https://www.fda.gov/combination-products/guidance-regulatory-information/postmarketing-safety-reporting-combination-products>); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820) for devices or current good manufacturing practices (21 CFR 4, Subpart A) for combination products; and, if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to <https://www.fda.gov/medical-devices/medical-device-safety/medical-device-reporting-mdr-how-report-medical-device-problems>.

For comprehensive regulatory information about medical devices and radiation-emitting products, including information about labeling regulations, please see Device Advice (<https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance>) and CDRH Learn (<https://www.fda.gov/training-and-continuing-education/cdrh-learn>). Additionally, you may contact the Division of Industry and Consumer Education (DICE) to ask a question about a specific regulatory topic. See the DICE website (<https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance/contact-us-division-industry-and-consumer-education-dice>) for more information or contact DICE by email ([DICE@fda.hhs.gov](mailto:DICE@fda.hhs.gov)) or phone (1-800-638-2041 or 301-796-7100).

Sincerely,

Gregory O'Connell  
Assistant Director  
DHT2C: Division of Coronary  
and Peripheral Intervention Devices  
OHT2: Office of Cardiovascular Devices  
Office of Product Evaluation and Quality  
Center for Devices and Radiological Health

Enclosure

## Indications for Use

510(k) Number (if known)  
K200744

Device Name  
Instylla Microcatheter

Indications for Use (Describe)

The Instylla Microcatheter is intended for use in small vessel or super selective anatomy for peripheral diagnostic and interventional procedures. The Instylla Microcatheter can be used for the infusion of diagnostic, embolic, or therapeutic materials into vessels.

Type of Use (Select one or both, as applicable)

Prescription Use (Part 21 CFR 801 Subpart D)

Over-The-Counter Use (21 CFR 801 Subpart C)

---

---

### CONTINUE ON A SEPARATE PAGE IF NEEDED.

---

---

This section applies only to requirements of the Paperwork Reduction Act of 1995.

**\*DO NOT SEND YOUR COMPLETED FORM TO THE PRA STAFF EMAIL ADDRESS BELOW.\***

The burden time for this collection of information is estimated to average 79 hours per response, including the time to review instructions, search existing data sources, gather and maintain the data needed and complete and review the collection of information. Send comments regarding this burden estimate or any other aspect of this information collection, including suggestions for reducing this burden, to:

Department of Health and Human Services  
Food and Drug Administration  
Office of Chief Information Officer  
Paperwork Reduction Act (PRA) Staff  
[PRASStaff@fda.hhs.gov](mailto:PRASStaff@fda.hhs.gov)

*"An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB number."*



## **K200744 - 510(k) Summary**

### **Submitter Information:**

Instylla, Inc.  
204 2<sup>nd</sup> Avenue  
Waltham, MA 02451

### **Contact Person:**

Jennifer Greer  
Regulatory Affairs Manager  
Phone: 585-260-2882  
E-mail: jennyg@instylla.com

### **Date Prepared:**

April 20, 2020

### **Subject Device:**

Proprietary Name:	Instylla Microcatheter
Common Name:	Catheter, Continuous Flush
Classification Name:	Continuous flush catheter (21 CFR 870.1210, Product Code KRA)
Device Classification:	Class II
Classification Panel:	Cardiovascular

### **Predicate Devices:**

Proprietary Name:	Instylla Microcatheter
Manufacturer:	Instylla, Inc.
510(k) Number:	K191731

### **Device Description:**

The Instylla Microcatheter includes a single lumen, multipurpose catheter intended for use in the peripheral vasculature. The basic operating principle is to advance the microcatheter through an outer guiding catheter and track coaxially over a steerable guidewire in order to access the treatment site. The microcatheter lumen is able to accommodate steerable guidewires that are

≤0.014in (0.36mm) in diameter. Once the target region has been accessed, the microcatheter can be used to deliver diagnostic, embolic, or therapeutic materials into vessels.

The microcatheter has a 1.7Fr (0.56mm) OD with a constant flexibility along its length. The ID of the microcatheter is 0.016in (0.41mm) along its length. The proximal end of the microcatheter incorporates a standard luer hub to enable the attachment of accessories, and a strain relief with a feature that allows for flexibility and securement inside a Tuohy-Borst adapter, for maintaining position inside a guiding catheter as needed. The microcatheter has a radiopaque marker at the distal tip to aid in fluoroscopic visualization. A 4Fr Tuohy-Borst with side-port adapter, a short extension adaptor, a long extension adapter and a duckbill check-valve are also included. The Instylla Microcatheter is available in 122cm, 142cm and 162cm usable lengths.

**Indications for Use:**

The Instylla Microcatheter is intended for use in small vessel or super selective anatomy for peripheral diagnostic and interventional procedures. The Instylla Microcatheter can be used for the infusion of diagnostic, embolic, or therapeutic materials into vessels.

The indications for use statement for the modified Instylla Microcatheter with check-valve connector remain unchanged from the cleared device.

**Comparison of Technological Characteristics to the Predicate Devices:**

The Instylla Microcatheter with check-valve connector is substantially equivalent in intended use and fundamental technological characteristics to the legally marketed predicate device. The below table summarizes the similarities in design and configuration of the Instylla Microcatheter with check-valve connector compared with the predicate devices.

<b>Attribute</b>	<b>Subject Device: Instylla Microcatheter with Check-Valve Connector</b>	<b>Predicate Device: Instylla Microcatheter (K191731)</b>
Indications for Use	The Instylla Microcatheter is intended for use in small vessel or super selective anatomy for peripheral diagnostic and interventional procedures. The Instylla Microcatheter can be used for the infusion of diagnostic, embolic, or therapeutic materials into vessels.	Same
Basic Design/ Components	Microcatheter, 4Fr Tuohy-Borst with Side Port Adaptor, Short Extension Adapter, Long Catheter Adapter and Duckbill Check-Valve Connector	Microcatheter, 4Fr Tuohy Borst with Side Port Adaptor, Short Extension Adapter, and Long Extension Adapter

Attribute	Subject Device: Instylla Microcatheter with Check- Valve Connector	Predicate Device: Instylla Microcatheter (K191731)
Catheter Material	Thermoplastic elastomers (PEBAX), PTFE (inner layer), platinum/iridium marker, polycarbonate (hub)	Same
Principle of Operation	Manually tracked over a steerable guidewire to access vasculature.	Same
Inner Diameter	0.016in (0.41mm)	Same
Outer Diameter	1.7Fr (0.56mm)	Same
Usable Lengths	122cm, 142cm, 162cm	Same
Guidewire Compatibility (Size)	0.014in (0.36mm)	Same
Radiopaque Marker	Yes	Same

The modified Instylla Microcatheter with check-valve connector only differs from the legally marketed predicate device in that it is packaged with one (1) duckbill check-valve connector. The intended use, indications, principle of operation, and technological characteristics remain identical between the subject and predicate device.

### Performance Data

Performance testing of the final, sterilized Instylla Microcatheter with check-valve connector included bench testing and functional testing to verify specifications related to the modification of the device. The following testing was repeated for the modified device:

- Accessory Compatibility and Functionality

The Instylla Microcatheter with check-valve connector met the predetermined acceptance criteria ensuring substantial equivalence to the predicate device. No new safety or performance issues were raised during testing.

### Biocompatibility Testing

A biocompatibility evaluation was conducted on the Instylla Microcatheter with check-valve connector in accordance with the FDA Guidance Document *Use of International Standard ISO 10993-1, "Biological Evaluation of Medical Devices Part 1: Evaluation and Testing within a Risk Management Process."* The following biocompatibility tests were successfully completed to demonstrate substantial equivalence of the modified Instylla Microcatheter with check-valve connector:

- Cytotoxicity

- Sensitization
- Irritation or Intracutaneous Toxicity
- Acute Systemic Toxicity
- Material-Mediated Pyrogenicity
- Hemolysis

### **Sterility**

The Instylla Microcatheter with check-valve connector is sterilized via a validated ethylene oxide (EO) process to a Sterility Assurance Level (SAL) of  $10^{-6}$ . The sterilization process was validated per ISO 11135 *Sterilization of health-care products — Ethylene oxide — Requirements for the development, validation and routine control of a sterilization process for medical devices*. A bacterial endotoxin test (BET), also known as the Limulus amoebocyte lysate (LAL) test, was also validated to establish that the microcatheter endotoxin level will be <20 endotoxin units (EU)/device.

### **Shelf Life**

The Instylla Microcatheter with check-valve connector has a 2-year shelf life. Shelf life studies have been conducted to demonstrate that the device maintains its performance and the packaging will maintain its sterile barrier over the entirety of the intended shelf life.

### **Clinical Performance Data**

The fundamental technological characteristics, indications for use, marker material, manufacturing and sterilization processes are the same as the predicate devices and therefore, no clinical studies were deemed necessary to demonstrate the safety and effectiveness of the subject device.

### **Conclusion**

Instylla has demonstrated that the Instylla Microcatheter with check-valve connector is substantially equivalent in fundamental design, function, device materials, packaging, sterilization, operating principle, intended use/indication for use and fundamental technology as the legally marketed predicate device, Instylla Microcatheter, which was cleared under 510(k) Premarket Notification K191731 on August 13, 2019.