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EN Description and Product details:

The Savoy International single use half-fold face mask FFP2 NR complies with the EN149 : 2009+A1 : :2009 standard. This half-fold face mask is designed to provide protection against liquid and solid aerosols.

Instruction for use :

The single use half-fold filtering face mask can be used as a protection device for everybody looking for a protection against liquid and solid aerosols. It is recommended to carefully read the instruction for use, and to have completed a formal training before wearing such half-fold filtering face mask. In case of question marks regarding this product use, contact either your supervisor or directly contact Savoy International. Before using the half-fold filtering face mask, wash your hands, remove the mask from its packaging right before using it. Before proceeding to half-fold filtering face mask sealing test, check status of the mask itself, rubber strips and expiry date. In case these elements doesn't comply with expectations, the half-fold filtering face mask shall be rejected.

Half-fold face mask sealing :

To check half-fold filtering face mask sealing, follow here after instructions/
- Place the mask over your face as described in the instruction manual
- Make the rubber band are in the right position
- Exhale strongly
- If leakages occur, follow here after instructions :
• Leakages around the nose, adjust the nose clip (cf. step #6)
• Leakages around the face, adjust the position of the mask in using the bottom tab, and tune rubber bands position (cf. step #3, #4 and #5). In case sealing doesn't comply with expectations, the half-fold filtering face mask shall be thrown away.

Warnings :

Beard hairs can significantly impact the half-fold filtering face mask sealing performances if they pass away from mask sealing area on the user face. Do not use the half-fold filtering face mask if it is damaged. Use the mask in a properly ventilated area, where the level of oxygen is high enough (>17%). Do not modify the half-fold filtering face mask. Do not use the half-fold filtering face mask in case it has been contaminated by blood or any other infectious element. The half-fold filtering face mask is building in latex free components.

Restrictions for use :

The half-fold filtering face mask doesn't protect the user against either gas or vapors. The half-fold filtering face mask protect the user against liquid and solid aerosols at a concentration level lower than 9 times the ELV (Exposure Limit Value). The half-fold filtering face mask doesn't provide the user with oxygen. The half-fold filtering face mask is not designed for a use in explosive environment. The half-fold filtering face mask is designed for a single use only and wearing time shall comply with either national or local regulations. The use of half-fold filtering face mask shall not exceed a workday period. Failure to follow instructions for use can reduce efficiency of the half-fold filtering face mask.

Compliance :

The half-fold filtering face mask complies with requirements defined by (EU) 2016/425 regulation, as well as EN149:2011+A1:2009 standard. The notified organization who has tested this product is APAVE: 8 rue Jean-Jacques Vernazza – ZAC Saumaty-Séon – BP193 13322 Marseille Cedex. Organization : 0082 – Completed modules : B and C2.

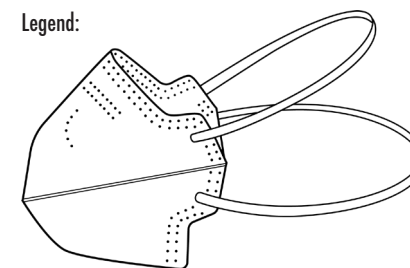
Storage :

The half-fold filtering face mask shall ideally be stored in a dry environment, in between +5°C and +40°C (+41°F and 113°F), <70% relative humidity. The half-fold filtering face mask shall be used before the expiry date indicated on the packaging.

For further assistance refer to savoy-international.com or email protect@savoy-international.com or call + 33 4 50 96 52 29 (France) or + 1 (561) 228 8779 (USA/UK).

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