
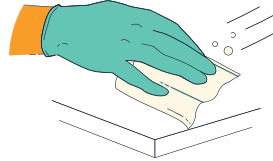


Important: Read all instructions thoroughly before you start.


Amazon COVID-19 Collection Kit | Questions: covid19testing@amazon.com


 Your test kit includes a biohazard bag, test tube and nasal swab. If you run into problems with your test kit, wipe down the area and discard kit in the waste bin at check-in station. Get a new kit.

1 Put on gloves. Wipe the station with disinfecting wipes. Then use hand sanitizer.




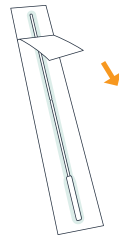
2 Remove the lid from the tube. Place the tube in the holder.

 Do not spill the fluid in the tube!

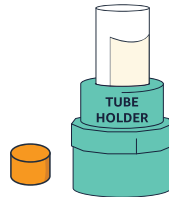


3 Open the swab pouch and remove from the package. Hold at the bottom.

 Do not touch the soft tip!




4 Lower mask below your nose. Keep your mouth covered.

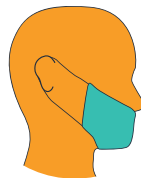


5 Swab first nostril.
Insert the soft end of the swab **1 inch**. Rub the swab **slowly** against the inside walls of your nostril at **least 4 times**.

This should take about 15 seconds.

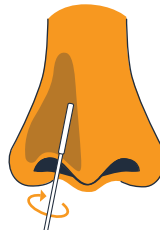



6 Repeat in other nostril.

Then place mask to cover your nose.

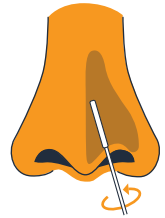


7 Place swab into the tube. Break at the indicator line by bending back and forth against the tube. Tightly recap the tube.

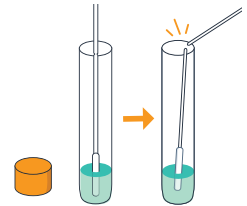


 Do not spill the fluid in the tube!

8 Place the tube into bag. Remove air and seal the bag.

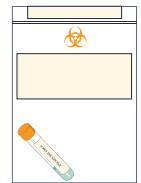


9 Wipe the station and chair with disinfecting wipes.

Put gloves and trash in the waste bin at your station. Keep your sample.

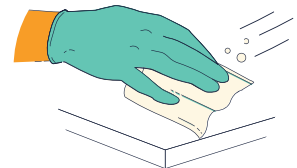

10 Put your sample in the collection bin as you exit.

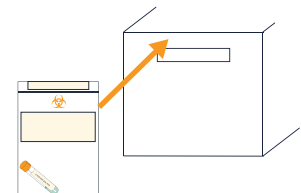


Amazon On-Site COVID-19 Test Collection Kit

Your kit includes:



Nasal swab

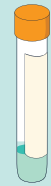

Tube with saline



Biohazard bag

Important reminders

- ✓ Keep your mask on while in the testing site. Your mask should cover your mouth at all times.
- ✓ Carefully follow all instructions. If you miss any testing steps your results may be inaccurate or result in an error.
- ✓ Collect a sample from both nostrils.
- ✓ The nasal swab tip should not touch anything except inside your nostril.
If the swab tip touches anything other than inside your nostril, discard your kit in the waste bin at check-in, clean the testing area with a disinfecting wipe, and request a new kit.
- ✓ Do not spill any of the fluid in the tube.
If you spill the fluid in the tube, discard your kit in the waste bin at check-in, clean the testing area with a disinfecting wipe, and request a new kit.

Questions?

There is a problem with my kit.

If you cannot complete the test based on the resources available at your testing station, request a new kit.

If you have questions, ask the flow monitor for help.

For additional questions, e-mail:
covid19testing@amazon.com



Do not touch or drink the fluid in the tube. If the fluid touches your eyes or skin, rinse with water.

1

Carefully read all instructions before you start.


If you have questions, ask the flow monitor for help.

2

Wipe the station and chair with disinfecting wipes. Then use hand sanitizer.



Do not remove gloves during sample collection.

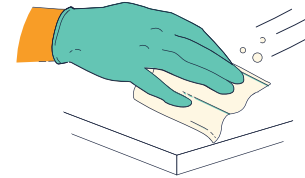


3

Remove the lid from the tube and place the tube in the holder.



Do not spill the fluid in the tube! If you spill the fluid, clean the area, discard your kit, and get a new kit.

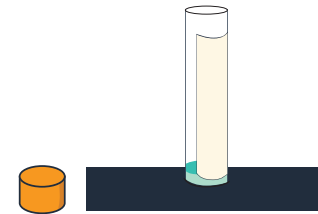


4

Open the swab pouch from the handle end. Remove swab from the package. Hold by the handle.



Do not touch the soft tip! If the swab tip touches anything other than inside your nostril, clean the area, discard your kit, and get a new kit.

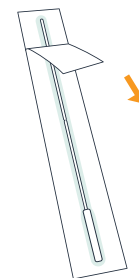


5

Lower face mask below your nose.




Do not remove mask. Keep your mouth covered while collecting sample.

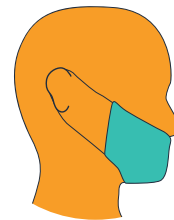

6

Swab first nostril.


Insert the soft tip of the swab **1/2 - 3/4 inch**. Rub the swab in a **slow circular motion** against the inside walls of your nostril at **least 4 times**. This should take about 15 seconds.

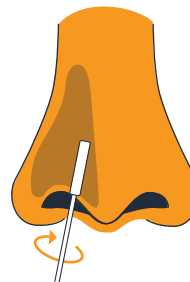

7

Repeat in other nostril.

Then place face mask back to cover your nose.

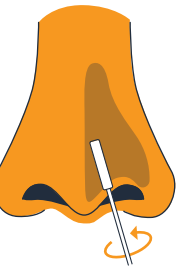


8

Place swab into the tube. Break off the swab handle.

To break the swab handle, hold the tube firmly. Bend the handle at the indicator line back and forth against the tube until it breaks. Tightly recap the tube.




Do not spill the fluid in the tube! If you spill the fluid, clean the area, discard your kit, and get a new kit.

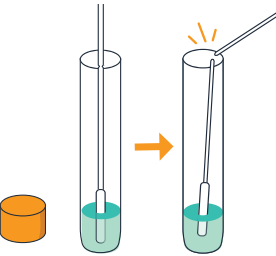

9

Place the tube into the bag, remove air, and seal the bag.


Refer to the **important reminders** in the left panel to ensure you collected your sample correctly. If you have not collected your sample correctly your test cannot be processed.

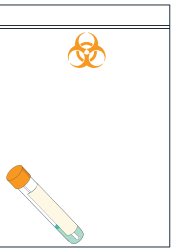

10

Wipe the station and chair with disinfecting wipes.


Put trash in the waste bin at your station. Keep your sample.

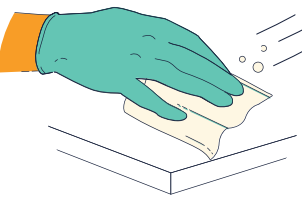


11

Put your sample in the collection bin as you exit.


Remove gloves and sanitize your hands as you exit.

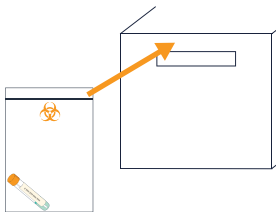

What's included

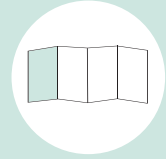

Instructions
for use

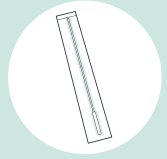

Nasal swab



Collection tube
with saline



Plastic bag

Important reminders

- You must register your sample collection kit to get results.
- You must wash (or sanitize) your hands before you collect your sample.
- Return your sample collection kit to your worksite within 24 hours of collecting your sample. Late samples cannot be processed.
- Sample collection takes less than 10 minutes.
- For frequently asked questions, visit amazon.com/testing-faq

Register your kit online



1 Visit [amazon.com](https://amazon.com/testing-faq)


Or, view the QR code with your camera app.

Sign in or create an account.



2 Confirm your test details

Make sure your name and date of birth are correct. **Enter the collection tube ID**, found on the side of your collection tube.

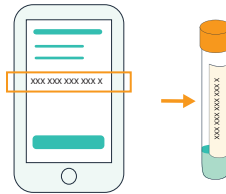


3 Press "Register kit"

Collect your sample **immediately** after registering. Sample collection takes less than 10 minutes.



You must register your kit to get results.

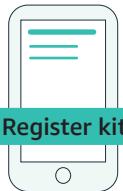

Watch the overview video during the register kit experience. Rewatch the video at: amazon.com/testing-faq

Collect your sample

Carefully read all instructions before collecting your sample.



Did you register your kit? If not, go back to steps 1–3.

We recommend collecting your sample while seated.

Do not touch the soft tip of the swab.

Do not touch or drink the fluid in the collection tube.

If the fluid touches eyes or skin, rinse with water.

If you spill the fluid or the swab touches any surface other than inside your nostrils, clean the area with your routine household surface disinfectant. Discard your kit components.

For frequently asked questions, visit amazon.com/testing-faq

4

Wash and dry your hands

Wash your hands with soap and warm water for **20 seconds**. Clean the surface of the area where you will collect your sample.

Use hand sanitizer if there is no sink.




5

Open the swab pouch

Peel back the tab on the handle end to remove the nasal swab from the pouch. **Hold by the handle**, do not touch the soft tip. Discard the swab pouch.



Do not touch the soft tip of the swab to any surface other than inside your nostrils.

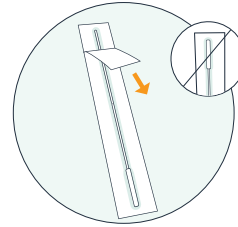


6

Swab first nostril

Insert the soft tip of the swab **1/2 - 3/4 inch** inside your nostril. Rub the swab in a **slow circular motion** against the inside walls of your nostril **at least 4 times**.


This should take about 15 seconds.

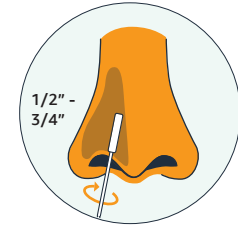

7

Repeat in other nostril

Repeat the previous step in your other nostril using the **same swab**.

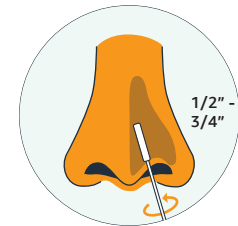


continued on back

8 Put swab in collection tube


While holding the swab, **open the tube**. Put the tube cap with the flat top down on a clean surface nearby. Put the **swab soft tip down** in the tube.

⚠ Do not spill the fluid in the tube!
If you spill the fluid, clean the area and discard your kit components.

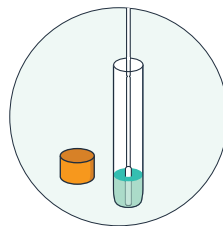

9 Break off the swab handle


To break off the swab handle, hold the tube firmly. Bend the swab handle at the indicator line back and forth against the collection tube until it breaks. Discard the swab handle.

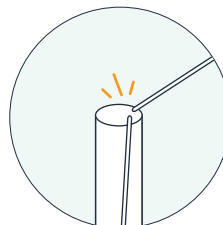

10 Recap the tube

Make sure the cap is on tight, so that no fluid can spill out. If the tube leaks we cannot process your sample.


Wash your hands (or use hand sanitizer).

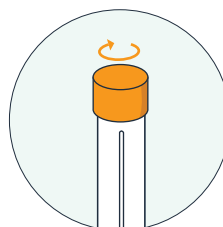

11 Put tube inside the plastic bag

Put the sealed tube inside the plastic bag. Release excess air from the bag and seal it shut.


Clean the surface area with a disinfectant.

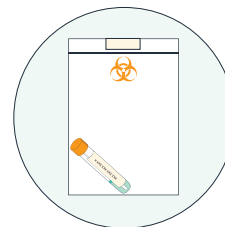

12 Checklist before returning your bag


- ☐ I registered my kit (if not, go back to steps 1-3).
- ☐ I collected a sample from both nostrils.
- ☐ The nasal swab only touched my nostrils.
- ☐ I did not spill the fluid in the tube.

If you have not checked all of the boxes, your test cannot be processed. Discard your kit components and get a new kit.

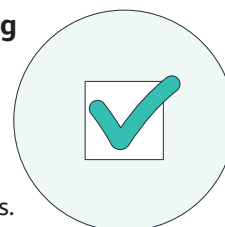

13 Drop off your bag at your worksite

Within 24 hours drop off your sealed bag at your designated worksite. Once your sample has been sent to the lab, you will receive updates via e-mail/SMS.

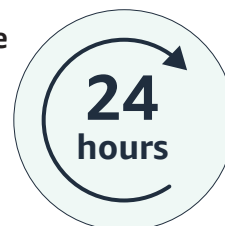

Questions?

Frequently asked questions
[amazondx.com/testing-faq](https://www.amazon.com/testing-faq)

E-mail
covid19testing@amazon.com

410 Terry Ave N, Seattle, WA 98109

For In Vitro Diagnostic Use Only. For Emergency Use Authorization Only.
Rx Only. For Use by Individuals 18 Years or Older.

This product has not been FDA cleared or approved but has been authorized for emergency use by FDA under an EUA for use by authorized laboratories. This product has been authorized only for the detection of nucleic acid from SARS-CoV-2, not for any other viruses or pathogens.

The emergency use of this product is only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of in vitro diagnostics for detection and/or diagnosis of COVID-19 under Section 564(b)(1) of the Federal Food, Drug and Cosmetics Act, 21 U.S.C. § 360bbb-3(b)(1), unless the declaration is terminated or authorization is revoked sooner.

Amazon COVID-19 Test Collection Kit

Instructions for use

