

Front

Back

Left

FDA AUTHORIZED



Detect™ Covid-19 Test

 Requires a Detect Hub (device sold separately).

 A smartphone with the Detect App (free) is required to use this test.

HOME TEST

For use under FDA Emergency Use Authorization (EUA) only. For individuals age 14+ for self testing. For children age 2+ when tested by an adult caregiver.

For testing individuals suspected of Covid-19, a single test may be used. If you have no reason to suspect Covid-19, you will need at least two tests per person. You may need to purchase additional tests to perform serial (repeat) testing.

PCR-quality.

Results in 1 hour.


Nothing to mail.

The Detect™ Covid-19 Test is a fast, easy, molecular test that helps diagnose a current Covid-19 infection.


Use the Detect App to follow the step-by-step video guide and to get your results.


Visit detect.com/app to see the list of compatible devices and download the app.


This is a molecular test for the detection of nucleic acids from SARS-CoV-2, not for any other viruses or pathogens. Single use only.


 Store at temperatures 59-86°F (15 - 30°C) in original sealed packaging.


Test includes

1 × Reader

1 × Detect Cap

1 × Test Tube

1 × Dropper

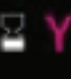
1 × Swab

Keep out of reach of children. Small parts are a choking hazard. Follow directions for use. Do not ingest. Avoid contact with skin and eyes.

detect.com
+1 (855) 322-3692
support@detect.com


Detect Covid-19

S/N: #####

Use by:  YYYY-MM-DD

Ref: 12345

Lot: 12345678



Made in Switzerland

PN: 21296_REV: A

Right

Top

Inside Lid


PCR-QUALITY. RESULTS IN 1 HOUR. NOTHING TO MAIL.


TEAR TO OPEN


Detect Covid-19


HOME TEST

Bottom

 Do not use if package is damaged.

 Electronic Instructions For Use available in the Detect App.


 PAPER BOX MULTILAYER POUCH

 8 60006 57460 4

Swab Made in Taiwan
Reader Made in Switzerland
Dropper Made in the United Kingdom

The Detect™ Covid-19 Test has not been FDA cleared or approved but has been authorized by FDA under Emergency Use Authorization (EUA) only. This product has been authorized only for the detection of nucleic acids from SARS-CoV-2, not for any other viruses or pathogens.

The emergency use of this product is only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of in vitro diagnostics for detection and/or diagnosis of Covid-19 under Section 564(b)(1) of the Federal Food, Drug and Cosmetic Act, 21 U.S.C. § 360bbb3(b)(1), unless the declaration is terminated or authorization is revoked sooner.

 Detect, Inc.
530 Old Whitfield St.
Guilford, CT 06437

IVD

Getting Started

01 . Have a Detect™ Hub ready and plugged in. The Detect Hub is sold separately and is required to complete the test.

02 . Use the Detect App and follow the instructions. The app will guide you through the test with step-by-step video instructions and help you interpret your test results. Follow all instructions in the app carefully.

03 . Prepare to perform the test on a flat, clean surface. Make sure to use the test within 1 hour of opening.


Get the app at detect.com/app

Activation

FOR TEST ACTIVATION ONLY. NOT FOR APP DOWNLOAD.


Activation Code


Scan this QR code when prompted by the app, or manually enter the serial number below.




AAAA - AAAA - AAAA - AAAA - AAAA - AAAA - AAAA

Need help?


 +1 (855) 322-3692

 support@detect.com

Front



Requires a Detect Hub
(device sold separately)



A smartphone with the Detect App
(free) is required to use this test.


Detect Covid-19 Test

2 - PACK

HOME TEST

FDA AUTHORIZED

For use under FDA Emergency Use Authorization (EUA) only. For individuals age 14+ for self testing. For children age 2+ when tested by an adult caregiver. For serial testing of individuals without reasons to suspect Covid-19. For testing individuals suspected of Covid-19, a single test may be used.

Electronic Instructions For Use
available in the Detect App.

Back

Molecular test for home use.

PCR-quality results in 1 hour.

Download the Detect™ App to follow the step-by-step video guide and to get your results.

Visit detect.com/app to do so and see the list of compatible devices.

Contains 2 Detect Covid-19 Tests, each including:


1 × Swab

1 × Detect Cap

1 × Reader

1 × Test Tube

1 × Dropper

Store at temperatures 59-86°F
(15-30°C) in original sealed packaging.

Keep out of reach of children. Small parts are a choking hazard. Follow directions for use. Do not ingest. Avoid contact with skin and eyes.

Do not store the test where temperatures are very cold (like the refrigerator) or very warm (like a car or in direct sunlight).

If you are suspected of Covid-19 (symptomatic or exposed), you can use a single test. If you have no reason to suspect Covid-19, testing must be performed twice over three days with at least 24 hours (and no more than 48 hours) between tests.


detect.com
+1 (855) 322-3692
support@detect.com

Top


MOLECULAR TEST FOR HOME USE, PCR-QUALITY RESULTS IN 1 HOUR

DETECT.COM

Left


Do not use if package is damaged.

PN: 21359_REV: A




Detect, Inc.
530 Old Whitfield St.
Guilford, CT 06437


IVD

OTC

The Detect™ Covid-19 Test has not been FDA cleared or approved but has been authorized by FDA under Emergency Use Authorization (EUA) only. This product has been authorized only for the detection of nucleic acids from SARS-CoV-2, not for any other viruses or pathogens.

The emergency use of this product is only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of in vitro diagnostics for detection and/or diagnosis of Covid-19 under Section 564(b)(1) of the Federal Food, Drug and Cosmetic Act, 21 U.S.C. § 360bbb3(b)(1), unless the declaration is terminated or authorization is revoked sooner.

PAPER
BOX

ENVIRONMENTAL
POUCH

8 60006 57462 8

Dropper made in the United Kingdom, Reader made in Switzerland, Swab made in Taiwan

Right

Detect Covid-19

DETECT.COM