



Canon Medical Systems Corporation
% Orlando Tadeo, Jr.
Sr. Manager, Regulatory Affairs
Canon Medical Systems USA
2441 Michelle Drive
TUSTIN CA 92780

April 5, 2022

Re: K220335

Trade/Device Name: Vantage Fortian/Orian 1.5T, MRT-1550, V8.0 with AiCE
Reconstruction Processing Unit for MR

Regulation Number: 21 CFR 892.1000

Regulation Name: Magnetic resonance diagnostic device

Regulatory Class: Class II

Product Code: LNH

Dated: February 3, 2022

Received: February 7, 2022

Dear Orlando Tadeo:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. Although this letter refers to your product as a device, please be aware that some cleared products may instead be combination products. The 510(k) Premarket Notification Database located at <https://www.accessdata.fda.gov/scripts/cdrh/cfdocs/cfpmn/pmn.cfm> identifies combination product submissions. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part

801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803) for devices or postmarketing safety reporting (21 CFR 4, Subpart B) for combination products (see <https://www.fda.gov/combination-products/guidance-regulatory-information/postmarketing-safety-reporting-combination-products>); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820) for devices or current good manufacturing practices (21 CFR 4, Subpart A) for combination products; and, if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to <https://www.fda.gov/medical-devices/medical-device-safety/medical-device-reporting-mdr-how-report-medical-device-problems>.

For comprehensive regulatory information about medical devices and radiation-emitting products, including information about labeling regulations, please see Device Advice (<https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance>) and CDRH Learn (<https://www.fda.gov/training-and-continuing-education/cdrh-learn>). Additionally, you may contact the Division of Industry and Consumer Education (DICE) to ask a question about a specific regulatory topic. See the DICE website (<https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance/contact-us-division-industry-and-consumer-education-dice>) for more information or contact DICE by email (DICE@fda.hhs.gov) or phone (1-800-638-2041 or 301-796-7100).

Sincerely,

For

Thalia T. Mills, Ph.D.
Director
Division of Radiological Health
OHT7: Office of In Vitro Diagnostics
and Radiological Health
Office of Product Evaluation and Quality
Center for Devices and Radiological Health

Enclosure

Indications for Use

510(k) Number (if known)

K220335

Device Name

Vantage Fortian/Orian 1.5T, MRT-1550, V8.0 with AiCE Reconstruction Processing Unit for MR

Indications for Use (Describe)

Vantage Fortian/Orian 1.5T systems are indicated for use as a diagnostic imaging modality that produces cross-sectional transaxial, coronal, sagittal, and oblique images that display anatomic structures of the head or body. Additionally, this system is capable of non-contrast enhanced imaging, such as MRA.

MRI (magnetic resonance imaging) images correspond to the spatial distribution of protons (hydrogen nuclei) that exhibit nuclear magnetic resonance (NMR). The NMR properties of body tissues and fluids are:

- Proton density (PD) (also called hydrogen density)
- Spin-lattice relaxation time (T1)
- Spin-spin relaxation time (T2)
- Flow dynamics
- Chemical Shift

Depending on the region of interest, contrast agents may be used. When interpreted by a trained physician, these images yield information that can be useful in diagnosis.

Type of Use (Select one or both, as applicable)

Prescription Use (Part 21 CFR 801 Subpart D)

Over-The-Counter Use (21 CFR 801 Subpart C)

CONTINUE ON A SEPARATE PAGE IF NEEDED.

This section applies only to requirements of the Paperwork Reduction Act of 1995.

DO NOT SEND YOUR COMPLETED FORM TO THE PRA STAFF EMAIL ADDRESS BELOW.

The burden time for this collection of information is estimated to average 79 hours per response, including the time to review instructions, search existing data sources, gather and maintain the data needed and complete and review the collection of information. Send comments regarding this burden estimate or any other aspect of this information collection, including suggestions for reducing this burden, to:

Department of Health and Human Services
Food and Drug Administration
Office of Chief Information Officer
Paperwork Reduction Act (PRA) Staff
PRASStaff@fda.hhs.gov

"An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB number."

510(k) SUMMARY

This summary of 510(k) safety and effectiveness information is being submitted in accordance with the requirements of Safe Medical Device Act 1990 and 21 CFR § 807.92

1. CLASSIFICATION and DEVICE NAME

Classification Name:	Magnetic Resonance Diagnostic Device
Regulation Number:	90-LNH (Per 21 CFR § 892.1000)
Trade Proprietary Name:	Vantage Fortian/Orian 1.5T, MRT-1550, V8.0 with AiCE Reconstruction Processing Unit for MR
Model Number:	MRT-1550

2. SUBMITTER'S NAME

Canon Medical Systems Corporation
1385 Shimoishigami
Otawara-Shi, Tochigi-ken, Japan 324-8550

3. OFFICIAL CORRESPONDENT

Naofumi Watanabe
Senior Manager, Regulatory Affairs and Vigilance
Canon Medical Systems Corporation

4. CONTACT PERSON, U.S. AGENT and ADDRESS

Contact Person

Orlando Tadeo, Jr.
Sr. Manager, Regulatory Affairs
Canon Medical Systems USA, Inc.
2441 Michelle Drive, Tustin, CA 92780
Phone: (714) 669-7459
Fax: (714) 730-1310
E-mail: otadeo@us.medical.canon

Official Correspondent/U.S. Agent

Paul Biggins
Senior Director, Regulatory Affairs
Canon Medical Systems USA, Inc.
2441 Michelle Drive, Tustin, CA 92780
Phone: (714) 730-7808
Fax: (714) 730-1310
E-mail: pbiggins@us.medical.canon



5. MANUFACTURING SITE

Canon Medical Systems Corporation
1385 Shimoishigami
Otawara-shi, Tochigi 324-8550, Japan

6. ESTABLISHMENT REGISTRATION

9614698

7. DATE PREPARED

February 2, 2022

8. DEVICE NAME

Vantage Fortian/Orian 1.5T, MRT-1550, V8.0 with AiCE Reconstruction Processing Unit for MR

9. TRADE NAME

Vantage Fortian/Orian 1.5T, MRT-1550, V8.0 with AiCE Reconstruction Processing Unit for MR

10. CLASSIFICATION NAME

Magnetic Resonance Diagnostic Device (MRDD)

11. CLASSIFICATION PANEL

Radiology

12. DEVICE CLASSIFICATION

Class II (per 21 CFR 892.1000, Magnetic Resonance Diagnostic Device)

13. PRODUCT CODE

90-LNH

14. PREDICATE DEVICE

Predicate Device (system): Vantage Fortian 1.5T, MRT-1550, V8.0 with AiCE Reconstruction Processing Unit for MR (K213305)

	Predicate Device	Reference Device (Previously cleared model)	Reference Device (Previously cleared model)	Reference Device (Previously cleared model)
System	Vantage Fortian 1.5T, MRT-1550, V8.0 with AiCE Reconstruction Processing Unit for MR	Vantage Orian 1.5T, MRT-1550, V7.0 with AiCE Reconstruction Processing Unit for MR	Vantage Orian 1.5T, MRT-1550, V7.0 with AiCE Reconstruction Processing Unit for MR	Vantage Orian 1.5T, MRT-1550, V6.0 with AiCE Reconstruction Processing Unit for MR
Marketed By	Canon Medical Systems USA, Inc.	Canon Medical Systems USA, Inc.	Canon Medical Systems USA, Inc.	Canon Medical Systems USA, Inc.
510(k) Number	K213305	K211633	K203053	K202210
Clearance Date	December 1, 2021	July 22, 2021	December 2, 2020	September 22, 2020

15. REASON FOR SUBMISSION

Modification of a cleared device

16. SUBMISSION TYPE

Traditional 510(k) Premarket Notification

17. DEVICE DESCRIPTION

The Vantage Fortian (Model MRT-1550/ WK, WM, WO, WQ)/Vantage Orian (Model MRT-1550/ A3, A4, A7, A8) is a 1.5 Tesla Magnetic Resonance Imaging (MRI) System. These Vantage Fortian/Orian models use 1.4 m short and 4.1 tons light weight magnet. They include the Canon Pianissimo™ Σ and Pianissimo Zen technology (scan noise reduction technology). The design of the gradient coil and the whole-body coil of these Vantage Fortian/Orian models provide the maximum field of view of 55 x 55 x 50 cm and include the standard gradient system.

The Vantage Orian (Model MRT-1550/ UC, UD, UG, UH, UK, UL, UO, UP, AK, AL, AO, AP, Upgrade only: A3, A4, A7, A8, AC, AD, AG, AH) is a 1.5 Tesla Magnetic Resonance Imaging (MRI) System. The Vantage Orian models MRT-1550/ UC, UD, UG, UH, UK, UL, UO, UP, Upgrade only: A3, A4, A7, A8 use 1.4 m short and 4.1 tons light weight magnet while the Vantage Orian models MRT-1550/ AK, AL, AO, AP, Upgrade only: AC, AD, AG, AH use 1.4 m short and 3.8 tons light weight magnet. All of the aforementioned models include the Canon Pianissimo™ and Pianissimo Zen technology (scan noise reduction technology). The design of the gradient coil and the whole-body coil of these Vantage Orian models provide the maximum field of view of 55 x 55 x 50 cm. The Model MRT-1550/ UC, UD, UG, UH, UK, UL, UO, UP, AK, AL, AO, AP includes the XGO gradient system. The Model MRT-1550/ A3, A4, A7, A8, AC, AD, AG, AH include the standard gradient system.

This system is based upon the technology and materials of previously marketed Canon Medical Systems MRI systems and is intended to acquire and display cross-sectional transaxial, coronal, sagittal, and oblique images of anatomic structures of the head or body. The Vantage Fortian/Orian MRI System is comparable to the current 1.5T Vantage Fortian MRI System (K213305), cleared December 1, 2021 with the following modifications.

18. SUMMARY OF CHANGE(S)

This submission is to report the following changes:

Summary of Software Changes:

- **Iterative Motion Correction (IMC) body region expansion:** IMC reduces motion artifact by correcting k-space data based on detecting the amount of motion during scan. Applying IMC in the brain corrects for rigid motion, applying IMC in the C-spine corrects for both rigid and non-rigid motion.
- **Exsper Sequence Expansion:** Exsper is available in FE2D and SE2D in addition to SE-EPI(DWI) and FSE2D
- **2D-RMC Sequence Expansion:** 2D-RMC is available in FSE2D and FASE2D in addition to FASE3D, FFE3D and SEEPI2D.

Summary of Accessory Changes:

- **Ceiling Camera feature expansion:** Patient orientation detection function and Patient anatomy position detection function for projector positioning are added.

19. SAFETY PARAMETERS

Item	Subject Device: Vantage Fortian/Orian 1.5T, MRT-1550, V8.0	Predicate Device: Vantage Fortian 1.5T, MRT-1550, V8.0	Notes
Static field strength	1.5T	1.5T	Same
Operational Modes	Normal and 1st Operating Mode	Normal and 1st Operating Mode	Same
i. Safety parameter display	SAR, dB/dt	SAR, dB/dt	Same
ii. Operating mode access requirements	Allows screen access to 1st level operating mode	Allows screen access to 1st level operating mode	Same
Maximum SAR	4W/kg for whole body (1st operating mode specified in IEC 60601-2-33: 2010+A1:2013+A2:2015)	4W/kg for whole body (1st operating mode specified in IEC 60601-2-33: 2010+A1:2013+A2:2015)	Same
Maximum dB/dt	1st operating mode specified in IEC 60601-2-33: 2010+A1:2013+A2:2015	1st operating mode specified in IEC 60601-2-33: 2010+A1:2013+A2:2015	Same
Potential emergency condition and means provided for shutdown	Shutdown by Emergency Ramp Down Unit for collision hazard for ferromagnetic objects	Shutdown by Emergency Ramp Down Unit for collision hazard for ferromagnetic objects	Same

20. IMAGING PERFORMANCE PARAMETERS

No change from the previous predicate submission, K213305.

21. INDICATIONS FOR USE

Vantage Fortian/Orian 1.5T systems are indicated for use as a diagnostic imaging modality that produces cross-sectional transaxial, coronal, sagittal, and oblique images that display anatomic structures of the head or body. Additionally, this system is capable of non-contrast enhanced imaging, such as MRA.

MRI (magnetic resonance imaging) images correspond to the spatial distribution of protons (hydrogen nuclei) that exhibit nuclear magnetic resonance (NMR). The NMR properties of body tissues and fluids are:

- Proton density (PD) (also called hydrogen density)
- Spin-lattice relaxation time (T1)
- Spin-spin relaxation time (T2)
- Flow dynamics
- Chemical Shift

Depending on the region of interest, contrast agents may be used. When interpreted by a trained physician, these images yield information that can be useful in diagnosis.

22. SUMMARY OF DESIGN CONTROL ACTIVITIES

Risk Management activities for new software functionalities and pulse sequences are included in this submission. The test methods used are the same as those submitted in the previously cleared submission of the predicate device, Vantage Fortian 1.5T, MRT-1550, V8.0 (K213305). A declaration of conformity with design controls is included in this submission.

23. SAFETY

This device is designed and manufactured under the Quality System Regulations as outlined in 21 CFR § 820 and ISO 13485 Standards.

This device is based upon the same technologies, materials and software as the predicate device. Risk activities were conducted in concurrence with established medical device development standards and guidance. Additionally, testing was done in accordance with applicable recognized consensus standards published by the International Electrotechnical Commission (IEC) for medical devices and the National Electrical Manufacturers Association (NEMA):

LIST OF APPLICABLE STANDARDS

- ANSI AAMI ES60601-1:2005/(R)2012 and A1:2012
- IEC60601-1-2 (2014)
- IEC60601-1-6 (2010), Amd.1 (2013)
- IEC60601-2-33 (2010), Amd.1 (2013), Amd.2 (2015)
- IEC60825-1 (2007,2014)
- IEC62304 (2006), Amd.1 (2015)
- IEC62366-1 (2020)
- NEMA MS 1 (2008)
- NEMA MS 2 (2008)
- NEMA MS 3 (2008)
- NEMA MS 4 (2010)
- NEMA MS 5 (2018)

24. TESTING

IMC (Iterative Motion Correction) for head and C-spine was evaluated using volunteer images. Testing confirmed that IMC may reduce motion artifacts.

The patient orientation and patient anatomy position detection method using Ceiling camera function were evaluated using volunteers. It was confirmed that percentage of successful patient orientation detection and cases requiring no correction for successful patient anatomy position detection met predetermined acceptance criteria.

Software Documentation for a Moderate Level of Concern, per the FDA guidance document, "Guidance for the Content of Premarket Submissions for Software Contained in Medical Devices Document" issued on May 11, 2005, is also included as part of this submission.

25. SUBSTANTIAL EQUIVALENCE

Canon Medical Systems Corporation believes that the Vantage Fortian/Orian 1.5T, MRT-1550, V8.0 Magnetic Resonance Imaging (MRI) System is substantially equivalent to the previously cleared predicate device, Vantage Fortian 1.5T, MRT-1550, V8.0, referenced in this submission.

Canon Medical Systems Corporation believes that the changes incorporated into the Vantage Fortian/Orian 1.5T, MRT-1550, V8.0 are substantially equivalent to the previously cleared predicate device.

26. CONCLUSION

The modifications incorporated into the Vantage Fortian/Orian 1.5T, MRT-1550, V8.0 do not change the indications for use or the intended use of the device. Based on the indications for use, technological characteristics, and safety and performance testing, the subject device meets the requirements that are considered adequate for its intended use and is substantially equivalent in design, principles of operation and indications for use to the predicate device.