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**EN- Instructions for use of respiratory mask FFP2 NR D for single use**

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**EN- Spread apart the rims of the mask to open it fully.**

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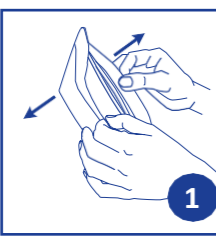
EN- Spread apart the rims of the mask to open it fully.

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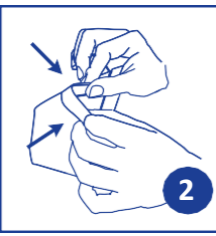
EN- Fold the nose bar to keep the respiratory mask open.

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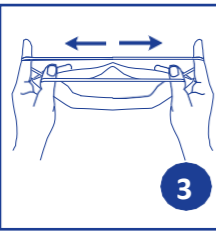
EN- Identify each elasticised string with your forefingers and thumbs.

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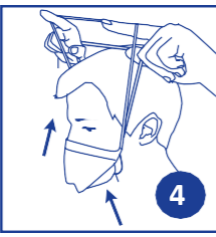
EN- Wedge the mask shaped as a hole under your chin, and pull the strings above your head, holding them with your thumbs and forefingers.

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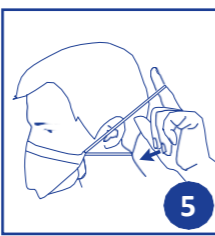
EN- Release the lower string and position it on the lower part of your neck.

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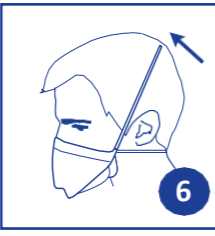
EN- Position the other string at the top of your head.

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EN- Adjust the nose bar, pressing the wings of your nose with your fingers firmly.

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EN- Position the mask until it fully adheres to your face.

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**EN- Description and indications:**

Respiratory Protection Masks FFP2 NR D for single use meet the requirements of standard EN149:2009.

These filtering masks are recommended to any person (industry personnel, health care professionals, patients, visitors...) looking for protection against liquid and solid sprays.

**Instructions for use:**

This mask may only be used by people who are trained to use it. Take the mask out of its wrapper only prior to immediate use. Before using it, ensure that it is intact and that elasticised strings are intact too

**To ensure that the respiratory mask tightly adheres to your face:**

- Cup your hands over the mask to block up its surface of filtration as much as possible, without moving it.

- Breathe out really strongly:

• if any air flows around your nose, readjust the nose bar as in picture 7.

• if any air flows out around the mask, carefully adjust the strings to the back.

In case tightness obtained is not satisfactory, do not enter the area where it is required to wear the mask.

Abbreviation "NR" : The use of the respiratory protection mask is limited to one working day.

Abbreviation "D" : The respiratory protection mask against particles meets the performance requirements of dolomite clogging.

**Warning:**

It is unlikely that tightness requirements are met if hairs from your beard stick out under the face seal.

Leave the premises if:

- you feel dizzy or feel any pain;

- it becomes hard to breathe.

Dispose of and replace the mask if:

- it is damaged;

- it seems to hamper your breathing;

- it is tainted by blood or any other infectious substance.

Dispose of the mask via the Infectious Hazard Care Activity Waste circuit.

**Use restrictions:**

This mask does not provide any protection against gases or fumes.

This half mask does not supply any oxygen.

Do not use in atmospheres which do not contain at least 17% oxygen.

Do not use in an explosive atmosphere.

	FFP2 NR D
Max. Concentration of Exposure	12 x OEL

OEL: Occupational Exposure Limit

This mask does not require any maintenance. It has been designed for one single continuous work session, i.e. for 8 hours as a maximum.

Any failure to comply with instructions and warnings concerning the use of this respiratory protection mask for the whole period of exposure may reduce its efficiency.

This product does not eliminate completely the risk of contracting diseases or infections.

Disposal of masks should be carried out if the expiry date marked on the overwrap is outdated.

**Conformities:**

This mask meets requirements defined by Regulation (EU) 2016/425.

Notified body used for EU type examination and manufacturing control of masks :

**APAVE SUDEUROPE SAS**  
8, rue Jean-Jacques Vernezza  
CS 60193  
13322 MARSEILLE, France (number:0082)

The EU declaration of conformity is available on our website : [www.macopharma.com](http://www.macopharma.com)

The respiratory protection mask FFP2 NR D do not contain any latex.

**Storage:**  
Store according to recommendations as displayed on the pack.

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