



July 8, 2022

Changsha Honchon Technology Co., LTD  
% Shanfeng Jiang  
Regulation Control Manager  
Guangzhou Junyi Information Technology Co., Ltd.  
Room 304, Building A, No. 62 Nanyun 2nd Road, Science Town  
Huangpu District, Guangzhou City, Guangdong 510663  
China

Re: K221051

Trade/Device Name: Dental Zirconia Blank & Dental Zirconia Pre-Shaded Blank  
Regulation Number: 21 CFR 872.6660  
Regulation Name: Porcelain Powder For Clinical Use  
Regulatory Class: Class II  
Product Code: EIH  
Dated: April 11, 2022  
Received: April 11, 2022

Dear Shanfeng Jiang:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. Although this letter refers to your product as a device, please be aware that some cleared products may instead be combination products. The 510(k) Premarket Notification Database located at <https://www.accessdata.fda.gov/scripts/cdrh/cfdocs/cfpmn/pmn.cfm> identifies combination product submissions. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal

statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803) for devices or postmarketing safety reporting (21 CFR 4, Subpart B) for combination products (see <https://www.fda.gov/combination-products/guidance-regulatory-information/postmarketing-safety-reporting-combination-products>); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820) for devices or current good manufacturing practices (21 CFR 4, Subpart A) for combination products; and, if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to <https://www.fda.gov/medical-devices/medical-device-safety/medical-device-reporting-mdr-how-report-medical-device-problems>.

For comprehensive regulatory information about medical devices and radiation-emitting products, including information about labeling regulations, please see Device Advice (<https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance>) and CDRH Learn (<https://www.fda.gov/training-and-continuing-education/cdrh-learn>). Additionally, you may contact the Division of Industry and Consumer Education (DICE) to ask a question about a specific regulatory topic. See the DICE website (<https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance/contact-us-division-industry-and-consumer-education-dice>) for more information or contact DICE by email ([DICE@fda.hhs.gov](mailto:DICE@fda.hhs.gov)) or phone (1-800-638-2041 or 301-796-7100).

Sincerely,

Srinivas Nandkumar, Ph.D.  
Director  
DHT1B: Division of Dental and ENT Devices  
OHT1: Office of Ophthalmic, Anesthesia,  
Respiratory, ENT and Dental Devices  
Office of Product Evaluation and Quality  
Center for Devices and Radiological Health

Enclosure

## Indications for Use

510(k) Number (if known)  
K221051

Device Name  
Dental Zirconia Blank & Dental Zirconia Pre-Shaded Blank

Indications for Use (Describe)

Dental Zirconia Blank & Dental Zirconia Pre-Shaded Blank are used for dental restorations using different CAD/CAM or manual milling machines. All blanks are processed through dental laboratories or by dental professionals.

Type of Use (Select one or both, as applicable)

Prescription Use (Part 21 CFR 801 Subpart D)

Over-The-Counter Use (21 CFR 801 Subpart C)

**CONTINUE ON A SEPARATE PAGE IF NEEDED.**

This section applies only to requirements of the Paperwork Reduction Act of 1995.

**\*DO NOT SEND YOUR COMPLETED FORM TO THE PRA STAFF EMAIL ADDRESS BELOW.\***

The burden time for this collection of information is estimated to average 79 hours per response, including the time to review instructions, search existing data sources, gather and maintain the data needed and complete and review the collection of information. Send comments regarding this burden estimate or any other aspect of this information collection, including suggestions for reducing this burden, to:

Department of Health and Human Services  
Food and Drug Administration  
Office of Chief Information Officer  
Paperwork Reduction Act (PRA) Staff  
[PRASStaff@fda.hhs.gov](mailto:PRASStaff@fda.hhs.gov)

*"An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB number."*

**Section 5 - 510(k) Summary****1. Submitter's Identifications**

Submitter's Name: Changsha Honchon Technology Co., LTD  
Address: Room 627-628, Building 3th, Jinqiao Market Cluster Area 2, Purui Road No.1555, Wangcheng Economic and Technological Development Zone, Changsha, Hunan, China  
Zip Code: 410200  
Contact Person: Ling Li  
Contact Title: General Manager  
Contact E-mail Address: tech@honchon.com  
Tel: +86-731-85529297

**2. Correspondent's Identifications**

Correspondent's Name: Guangzhou Junyi Information Technology Co., Ltd.  
Address: Room 304, Building A, No. 62 Nanyun 2nd Road, Science Town, Huangpu District, Guangzhou City, Guangdong, 510663, China  
ZIP Code: 510663  
Contact Person: Shanfeng Jiang  
Contact Title: Regulation Control Manager  
Contact E-mail Address: Jiang13620586569@126.com  
Tel: +86-20-82329549  
Fax: +86-20-82329549

**3. Name of the Device**

Device Classification Name: Powder, Porcelain  
Regulation Description: Porcelain powder for clinical use  
Trade Name: Dental Zirconia Blank & Dental Zirconia Pre-Shaded Blank  
Model: HS, HT, HT-C, HT-Plus, HTP-C, ST, ST-C, ST-Plus, STP-C, ST-ML, SHT, SHT-C, SHT-ML, 3D-ML, 3D-Plus-ML, 3D-Pro-ML, 4D-ML, 4D-Plus-ML, 4D-Pro-ML, UT, UT-C, UT-ML  
Regulation Medical Specialty: Dental  
Review Panel: Dental  
Product Code: EIH  
Regulation Number: 21 CFR 872.6660  
Device Classification: Class II

**4. The Predicate Devices**

Predicate device: K141724 Upcera Dental Zirconia Blank & Dental Zirconia Pre-Shaded Blank  
Liaoning Upcera Company Limited

## 5. Device Description

Dental Zirconia Blank are derived from zirconia powder that has been processed into their final shapes. These blanks are then being further fabricated into all-ceramic restorations such as crowns, bridges, veneers, inlay/onlay. The zirconia powder is composed of  $ZrO_2 + Y_2O_3 + HfO_2 + Al_2O_3$  with its composition conforms to ISO 13356, *Implants for surgery – Ceramic materials based on yttrium-stabilized tetragonal zirconia (Y-TZP)*. The performance of the dental blanks conforms to ISO 6872, *Dentistry – Ceramic materials*.

Dental Zirconia Pre-Shaded Blank are derived from the same Zirconia powder as the regular Dental Zirconia Blank with the addition of very small amount of inorganic pigments before the composite material is processed into their final net shapes. These blanks are then being further fabricated into all-ceramic restorations such as crowns, bridges, veneers, inlay/onlay. The purpose of the inorganic pigments is to generate the color on the prosthetic dental devices, after sintering at dental labs, that matches natural color of patient's teeth. The performance of the dental blanks conforms to ISO 6872, *Dentistry – Ceramic materials*.

## 6. Intended Use of Device

Dental Zirconia Blank & Dental Zirconia Pre-Shaded Blank are used for dental restorations using different CAD/CAM or manual milling machines. All blanks are processed through dental laboratories or by dental professionals.

## 7. Summary of Substantial Equivalence

**Table 1 Comparison to Predicate Device**

	<b>Proposed Device</b>	<b>Predicate device</b>	<b>Comparison</b>
<b>510k Number</b>		K141724	-----
<b>Product Code</b>	EIH	EIH	Same
<b>Proprietary Name</b>	Dental Zirconia Blank & Dental Zirconia Pre-Shaded Blank	Ucera Dental Zirconia Blank & Dental Zirconia Pre-Shaded Blank	-----
<b>Model</b>	HS, HT, HT-C, HT-Plus, HTP-C, ST, ST-C, ST-Plus, STP-C, ST-ML, SHT, SHT-C, SHT-ML, 3D-ML, 3D-Plus-ML, 3D-Pro-ML, 4D-ML, 4D-Plus-ML,	-----	-----

*Changsha Honchon Technology Co., LTD*

	4D-Pro-ML, UT, UT-C, UT-ML		
<b>Manufacturer</b>	Changsha Honchon Technology Co., LTD	Liaoning Upcera Company Limited	-----
<b>Indications for Use</b>	Dental Zirconia Blank & Dental Zirconia Pre-Shaded Blank are used for dental restorations using different CAD/CAM or manual milling machines. All blanks are processed through dental laboratories or by dental professionals.	Upcera Dental Zirconia Blank & Dental Zirconia Pre-Shaded Blank are used for dental restorations using different CAD/CAM or manual milling machines. All blanks are processed through dental laboratories or by dental professionals.	Same
<b>Basic design</b>	Blocks, and rods	Blocks, disc, and rod	Same
<b>Materials</b>	Regular: Zirconia (ZrO <sub>2</sub> + Y <sub>2</sub> O <sub>3</sub> + HfO <sub>2</sub> + Al <sub>2</sub> O <sub>3</sub> ≥99.0%)  Pre-shaded: Zirconia (ZrO <sub>2</sub> + Y <sub>2</sub> O <sub>3</sub> + HfO <sub>2</sub> + Al <sub>2</sub> O <sub>3</sub> ≥99.0%)  Inorganic pigments (Fe <sub>2</sub> O <sub>3</sub> , Er <sub>2</sub> O <sub>3</sub> and MnO <2.0%)	Regular: Zirconia (ZrO <sub>2</sub> + Y <sub>2</sub> O <sub>3</sub> + HfO <sub>2</sub> + Al <sub>2</sub> O <sub>3</sub> ≥99.0%)  Pre-Shaded: Zirconia (ZrO <sub>2</sub> + Y <sub>2</sub> O <sub>3</sub> + HfO <sub>2</sub> + Al <sub>2</sub> O <sub>3</sub> ≥98.0%)  Inorganic pigments (Fe <sub>2</sub> O <sub>3</sub> , Pr <sub>2</sub> O <sub>3</sub> , and Er <sub>2</sub> O <sub>3</sub> <2.0%)	Similar <sup>1</sup>
<b>Processing</b>	Sintering at temperature: > 1450°C	Sintering at temperature: > 1500°C	Similar <sup>2</sup>
<b>Dimension</b>	Various	Various	Same
<b>Single Use</b>	Yes	Yes	Same
<b>Color</b>	None, and Pre-shaded (for pre-shaded series)	None, and Pre-shaded (for pre-shaded series)	Same

<b>Sterile</b>	Non-sterile	Non-sterile	Same
----------------	-------------	-------------	------

The proposed device is essentially identical to the predicate devices in terms of indication for use, design between our device and the predicate devices. The minor differences are that as below:

Note 1: Zirconia ( $ZrO_2 + Y_2O_3 + HfO_2 + Al_2O_3$ ) of proposed device which is not less than 99.0% is higher than that of the predicate device. The three inorganic pigments of the proposed device contain  $Fe_2O_3$ ,  $Er_2O_3$  and  $MnO$ , while those of the predicate device are  $Fe_2O_3$ ,  $Pr_2O_3$ , and  $Er_2O_3$ . These are minor differences and the biocompatibility testing of the overall proposed device passed.

Note 2: Processing (Sintering at temperature) of the proposed device is slightly lower  $5^\circ C$  than that of the predicate device. This is minor difference and the performance testing of the proposed device passed.

### **8. Summary of Non-Clinical Testing**

Bench testing was performed per ISO 6872:2015 and internal procedures to ensure that the Dental Zirconia Blank & Dental Zirconia Pre-Shaded Blank met its specifications. All tests were verified to meet acceptance criteria. Test results on radioactivity, flexural strength, chemical solubility, linear thermal expansion coefficient, freedom from extraneous materials, uniformity, shrinkage factor, pre-sintered density, sintered density, and fracture toughness of the subject device are very similar to predicate device. Biocompatibility testing was performed to verify the equivalent safety of the materials that are used.

According to ISO 10993-1:2018 and ISO ISO7405:2018, we evaluated and conducted the compatibility test for the proposed device. The following table shows the biocompatibility testing results.

**Table 2 Biocompatibility testing**

Item	Proposed device	Result
Cytotoxicity (ISO 10993-5:2009)	Under the conditions of this study, the test article was non cytotoxic for 2 h and 24 h in the filter diffusion. Under the conditions of this study, the test article was None cytotoxic and was accepted in the agar diffusion . Under the conditions of this study, the test article has no potential toxicity to L-929 cells.	Pass
Oral Mucosa Irritation (ISO 10993-10:2010)	The test article showed no evidence of causing oral mucosa irritation in the Syrian hamsters.	Pass
Skin Sensitization (ISO 10993-10:2010)	The test article showed no evidence of causing delayed dermal contact sensitization in the guinea pig.	Pass

Subacute Systemic Toxicity (ISO 10993-11:2017)	The test result showed that the test article did not induce subacute systemic toxicity in rats under this condition .	Pass
Subchronic Systemic Toxicity (ISO 10993-11:2017)	There is no obvious histopathological difference in test group and control group. The structure of each organ in test group is normal, no abnormal histopathological changes in the table above was found.	Pass
Acute Systemic Toxicity (ISO 10993-11:2017)	The test article showed no evidence of causing acute system toxicity in the ICR mice.	Pass
In Vitro Mammalian Cell Gene Mutation (ISO 10993-3:2014)	Under the conditions of this study, the test article is considered non-mutagenic.	Pass
Micronucleus (ISO 10993-3:2014)	The test result showed that the extract of the test article did not induce the potential genotoxicity toxicity in ICR mice under this condition.	Pass
Bacterial Reverse Mutation (ISO 10993-3:2014)	Under the conditions of this study, the number of reverting colonies in the test article group is not equal to or greater that of the spontaneous control, so the test article have no potential mutagenesis.	Pass
Muscle Implant (ISO 10993-6:2016)	The test result showed that the test article did not induce local effects after implantation of biomaterials in rabbits under this condition.	Pass

Note: Testing were Performed on pre-shaded zirconia blank (UT-C) to cover the regular and pre-shaded zirconia blanks.

### **9. Clinical Test Conclusion**

No clinical study is included in this submission.

### **10. Conclusion:**

The conclusions drawn from the nonclinical tests demonstrate that subject device Dental Zirconia Blank & Dental Zirconia Pre-Shaded Blank performs as well as or better than the legally marketed predicate device K141724 Upcera Dental Zirconia Blank & Dental Zirconia Pre-Shaded Blank. Dental Zirconia Blank & Dental Zirconia Pre-Shaded Blank is substantial equivalent to the legally marketed predicate device K141724 Upcera Dental Zirconia Blank & Dental Zirconia Pre-Shaded Blank.