

Instructions for Use:

1. Open up the mask and make sure the nose bridge is on top and the interior of the mask is facing the user.
2. Place a loop around each ear, and pull the bottom of the mask to make sure the user's mouth and chin are covered.
3. Mold or pinch the nose bridge to the shape of the user's nose.
4. Adjust the elastic band and the adjustment tensioner to the appropriate felt strength of the elastic bands and ensure a close seal between the mask and the user's face.
5. Check the fit of the mask to the face:
 - a) Cover the mask with both hands to avoid affecting the position of the mask to the face.
 - b) Breathe in and out and if air leaks from the ledge of the mask please re-adjust the elastic band; if the user cannot get a good fit, repeat steps 1-4.

