

December 4, 2022

Jiangxi Handspro Products Solutions CO., LTD. % Boyle Wang Official Correspondent Shanghai Truthful Information Technology Co., Ltd. RM.1801,No.161, East Lujiazui Rd.,Pudong Shanghai, 200120 China

Re: K223004

Trade/Device Name: Disposable Nitrile Examination Gloves

Regulation Number: 21 CFR 880.6250

Regulation Name: Non-Powdered Patient Examination Glove

Regulatory Class: Class I, reserved

Product Code: LZA

Dated: September 29, 2022 Received: September 29, 2022

Dear Boyle Wang:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. Although this letter refers to your product as a device, please be aware that some cleared products may instead be combination products. The 510(k) Premarket Notification Database located at https://www.accessdata.fda.gov/scripts/cdrh/cfdocs/cfpmn/pmn.cfm identifies combination product submissions. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the <u>Federal Register</u>.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal

statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803) for devices or postmarketing safety reporting (21 CFR 4, Subpart B) for combination products (see https://www.fda.gov/combination-products/guidance-regulatory-information/postmarketing-safety-reporting-combination-products); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820) for devices or current good manufacturing practices (21 CFR 4, Subpart A) for combination products; and, if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to https://www.fda.gov/medical-device-problems.

For comprehensive regulatory information about medical devices and radiation-emitting products, including information about labeling regulations, please see Device Advice (https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance) and CDRH Learn (https://www.fda.gov/training-and-continuing-education/cdrh-learn). Additionally, you may contact the Division of Industry and Consumer Education (DICE) to ask a question about a specific regulatory topic. See the DICE website (https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance/contact-us-division-industry-and-consumer-education-dice">https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance/contact-us-division-industry-and-consumer-education-dice) for more information or contact DICE by email (DICE@fda.hhs.gov) or phone (1-800-638-2041 or 301-796-7100).

Sincerely,

Bifeng Qian -S

Bifeng Qian, M.D., Ph.D.
Assistant Director
DHT4B: Division of Infection Control
and Plastic Surgery Devices
OHT4: Office of Surgical
and Infection Control Devices
Office of Product Evaluation and Quality
Center for Devices and Radiological Health

Enclosure

DEPARTMENT OF HEALTH AND HUMAN SERVICES Food and Drug Administration

Indications for Use

510(k) Number (if known)

Form Approved: OMB No. 0910-0120

Expiration Date: 06/30/2023 See PRA Statement below.

K223004				
Device Name Disposable Nitrile Examination Gloves				
Indications for Use <i>(Describe)</i> The Disposable Nitrile Examination Gloves are disposable dev	vices intended for medical purposes that are worn on the			
examiner's hands to prevent contamination between patient and				
Type of Use (Select one or both, as applicable)				
Prescription Use (Part 21 CFR 801 Subpart D)	Over-The-Counter Use (21 CFR 801 Subpart C)			
CONTINUE ON A SEPARA	ATE PAGE IF NEEDED.			

This section applies only to requirements of the Paperwork Reduction Act of 1995.

DO NOT SEND YOUR COMPLETED FORM TO THE PRA STAFF EMAIL ADDRESS BELOW.

The burden time for this collection of information is estimated to average 79 hours per response, including the time to review instructions, search existing data sources, gather and maintain the data needed and complete and review the collection of information. Send comments regarding this burden estimate or any other aspect of this information collection, including suggestions for reducing this burden, to:

Department of Health and Human Services Food and Drug Administration Office of Chief Information Officer Paperwork Reduction Act (PRA) Staff PRAStaff@fda.hhs.gov

"An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB number."

510(k) Summary K223004

This summary of 510(k) is being submitted in accordance with 21 CFR 807.92.

1.0 Submitter's Information

Name: JIANGXI HANDSPRO PRODUCTS SOLUTIONS CO., LTD. Address: Chuangye Avenue, Fenglin Industrial Zone of Dean County,

Jiujiang City, Jiangxi, China.

Tel: +86-18616636918

Contact: Jun Yin

Designated Submission Correspondent

Name: Shanghai Truthful Information Technology Co., Ltd.

Address: Room 1801, No. 161 East Lujiazui Rd., Pudong, Shanghai

200120 ,China

Tel: +86-21-50313932
Contact: Mr. Boyle Wang
Email: Info@truthful.com.cn

Date of Preparation: Sep.29th,2022

2.0 <u>Device Information</u>

Trade name: Disposable Nitrile Examination Gloves

Common name: Patient Examination Gloves

Classification name: Non-powdered patient examination glove

Model(s): S, M, L, XL

Production code: LZA

Regulation number: 21CFR880.6250

Classification: Class I

Panel: General Hospital

3.0 Predicate Device Information

Manufacturer: GUANG DONG KINGFA SCI. & TECH.CO., LTD.

Device: Nitrile examination gloves

510(k) number: K203593

4.0 Indication for Use

The Disposable Nitrile Examination Gloves are disposable devices intended for medical purposes that are worn on the examiner's hands to prevent contamination between patient and examiner.

5.0 <u>Device Description</u>

The subject device is powder free nitrile examination gloves. The subject device is in blue color. It can be available in four specifications: S,M,L and XL. The subject device is non-sterile.

6.0 <u>Technological Characteristic Comparison Table</u>

Table1-General Comparison

Item	Subject Device Predicate (K223004) Device (K203593)		comparison
Product Code	LZA	LZA	same
Regulation No.	21CFR880.6250	21CFR880.6250	Same
Class	I	I	Same
Intended Use	The Disposable Nitrile Examination Gloves are disposable devices intended for medical purposes that are worn on the examiner's hands to prevent contamination between patient and examiner.	The nitrile examination glove is intended to be worn on the hands of examiners to prevent contamination between patient and examiner. This is a single-use, powder-free, non-sterile device.	Same
Material	Nitrile	Nitrile	Same
Powdered or Powered free	Powdered free	Powdered free	Same
Design Feature	Design Feature Ambidextrous		Same
Colorant	Colorant Blue		Same
Labeling Information	Single-use indication, powder free, device color, device name, glove size and quantity, non-Sterile	Single-use indication, powder free, device color, device name, glove size and quantity, non-Sterile	same

	Length:	Length:	same
	S:≥220	S:≥220	
	M/L/XL: ≥230	M/L/XL:≥230	
Dimensions(mm)	Width:	Width:	
Dimensions(mm)	S: 80±10	S: 80±10	
	M: 95±10	M: 95±10	
	L: 110±10	L: 110±10	
	XL: 120±10	XL: 120±10	
Thickness(mm)	Finger:≥0.05	Finger: ≥0.05	Same

		Palm: ≥0.05		Palm: ≥0.05		
	Before Aging	Tensile Strength	14MPa, min	Tensile Strength	14MPa, min	Same
		Ultimate Elongation	500% min	Ultimate Elongation	500% min	Same
Properties	After	Tensile Strength	14MPa, min	Tensile Strength	14MPa, min	Same
	Aging	Ultimate Elongation	400%min	Ultimate Elongation	400%min	Same
Freedom Hole		Be free from holes when tested in accordance with ASTMD5151 AQL=2.5		Be free from holes when tested inaccordance with ASTMD5151 AQL=2.5		same
Powder Content		Meet the requirements of ASTM D6124		Meet the requirements of ASTM D6124		Same
Biocompatibility		ISO 10993-10. Under the conditions of the study, not an irritant or a sensitizer			onditions of ot an irritant	Same
		ISO 10993-11. Under the condition of acute systemic toxicity test, the test article did not show acute systemic toxicity in vivo.		ISO 10993-11. Under the condition of acute systemic toxicity test, the test article did not show acute systemic toxicity in vivo.		same
		ISO 10993-5 Under conditions of the study, device extract is cytotoxic		ISO 10993-5 Under conditions of the study, device extract is cytotoxic		same

7.0 Summary of Non-clinical Testing

Non-clinical tests were conducted to verify that the proposed device met all design specifications. The test results demonstrated that the proposed device complies with the following standards:

ISO 10993-10:2010, Biological Evaluation of Medical Devices - Part 10: Tests For Irritation And Skin Sensitization.

ISO 10993-11:2017, Biological evaluation of medical devices - Part 11: Tests for systemic toxicity.

ASTM D6124-06 (Reapproved 2017), Standard Test Method for Residual Powder on Medical Gloves

ASTM D5151-19, Standard Test Method for Detection of Holes in Medical Gloves.

ASTM D6319-19, Standard Specification for Nitrile Examination Gloves for Medical Application.

Table 2 - Summary of non-clinical performance testing

Test	Purpose	Acceptance Criteria			Results
Method					
		Length(mm):		Length: S:>	
		S:≥220;		220/Pass;	
		M/L/XL:≥23	30;		M/L/XL: > 230/Pass
		Width(mm)	:		Width:
ASTM	Physical	S: 80±10;			S: 82-88 /Pass
D6319	Dimensions	M: 95±10;			M: 95-98/ Pass
D0319	Test	L: 110±10;			L: 106-111/ Pass
		XL: 120±10)		XL:111-115/ Pass
		Thickness	(mm):		Finger: 0.13-0.15/Pass
		Finger: ≥0.	05		Palm: 0.09-0.12/Pass
		Palm: ≥0.0	5		
ASTM	Watertightness	Meet the r	equirements of	0/125/Pass	
D5151	Test for	AQL 2.5			
	Detection of				
	Holes				
ASTM	Powder	Meet the re	equirements of A	S:0.13 mg/glove;	
D6124	Content	2.0mg/glove		M:0.13 mg/glove;	
					L:0.16 mg/glove;
					XL:0.11 mg/glove
		Before	Tensile	≥14MPa	20-34/Pass;
		Aging	Strength		
			Ultimate	≥500%	521-570/Pass;
ASTM	Physical		Elongation		
D412	properties	After	Tensile	≥14MPa	15-25/Pass;
		Aging	Strength		
			Ultimate	≥400%	470-545/Pass;
			Elongation		
ISO 10993-5	Cytotoxicity	Non- cytotoxic			Under the condition of the study, the device extract showed cytotoxic / fail.

ISO 10993-11	Acute systemic toxicity	Non- acute systemic toxicity	Under conditions of the study, did not show acute systemic toxicity in vivo / Pass
ISO	Irritation	Non-irritating	Under the conditions
10003-10			of the study not an

ISO	Irritation	Non-irritating	Under the conditions	
10993-10			of the study, not an	
			irritant/ Pass	
ISO	Sensitization	Non-sensitizing	Under conditions of	
10993-10			the study, not a	
			sensitizer./ Pass	

8.0 Summary of Clinical Testing

Clinical testing is not needed for this device.

9.0 Conclusion

The conclusions drawn from the nonclinical tests demonstrate that the subject device Disposable Nitrile Examination Gloves, is as safe, as effective, and performs as well as or better than the legally marketed predicated device.