



IVD  
EverlyWell  
Austin, TX

Email: covid19@everlywell.com  
Toll Free: +1 855-463-1670  
Barcode: 8 51044 00842 8



CLIA-Certified  
Lab Partners



# COVID-19 Test Home Collection Kit DTC

Physician-reviewed test results, fast and easy-to-read results, all from the comfort of your home.



Register Kit



Collect Sample  
& Ship



Receive Fast  
Online Results

IVD For Use Under Emergency Use Authorization



Overnight shipping  
to lab included



# COVID-19 Test Home Collection Kit DTC

Physician-reviewed test results, fast and easy-to-read results, all from the comfort of your home.

[www.everlywell.com](http://www.everlywell.com)



Warnings and Limitations  
Indicated for use in individuals age 18+ (self-collected), 16+ (self-collected under adult supervision), or 2+ (collected with adult assistance). Check everlywell.com/states to check what states this test is valid in. Requires access to the internet and an email account. Store kit at room temperature (4-30°C) until sample collection. This product has not been FDA cleared or approved, but has been authorized for emergency use by FDA under an EUA. This product has been authorized only for the home collection and maintenance of nasal swab specimens as an aid in detection of nucleic acid from SARS-CoV-2, not for any other viruses or pathogens. The emergency use of this product only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of medical devices under Section 564(b)(1) of the Federal Food, Drug and Cosmetic Act, 21 U.S.C. § 360bbb-3(b)(1), unless the declaration is terminated or authorization is revoked sooner.

- Contents
- Tube and Swab
  - Alcohol Prep Pad
  - Kit ID Stickers
  - Biohazard Bag with Absorbent Sheet
  - Return Box
  - Return Envelope
  - Return Label
  - Instructions and Fact Sheet for Individuals

### Register

Register your kit online at [everlywell.com/register](http://everlywell.com/register) by entering the unique kit ID found within this package.

### Collect

You will collect your sample at home using the collection instructions included in this package and then mail your sample directly to our CLIA-certified lab partner.

If positive results are shown, you will be contacted by our independent physician network for next steps.

### Review

### Why EverlyWell?

**Convenient**  
Complete your sample collection from home.

**Save Time + Money**  
Your time is valuable, own your health from home.

**Actionable**  
Insights you can understand with tailored advice on your next steps.

**Authorized**  
CLIA-certified labs. Physician-reviewed results.



Your Sample Will Be Tested For:  
The presence or absence of RNA from SARS-CoV-2 virus

# Return Instructions

Double-check that the sample is registered at [everlywell.com/register](https://everlywell.com/register) and that the completed ID sticker is affixed to the sample tube.  
If you are under 18, please ask your parent or guardian for assistance with the packaging and return shipment of your specimen.

- 1



**Wash your hands**  
Wash your hands thoroughly with soap and water for 20 seconds.
- 2



**Get return box ready**  
Assemble the box (insert flaps or pop open) and close one end.
- 3



**Prepare sample**  
Place the biohazard bag inside the box. The box provides extra protection during transit.
- 4



**Put box in mailer**  
With clean hands, place the shipping box into the prepaid mailer. Apply the shipping label to the front of the mailer.  
**Do not cover the UN3373 label.**
- 5



**Disinfect the mailer**  
Use the provided alcohol prep pad to disinfect the mailer, avoiding the address label, before completing the final shipping step.
- 6



**Ship**  
Deliver the pre-paid, pre-addressed return pack to a UPS drop box. It's important to drop off the sample on the **same day the sample is collected and before the last pickup** for overnight shipping. **Do not deliver your sample to a drop box on the weekend.**

909-554-1900

\*This product has not been FDA cleared or approved but has been authorized for emergency use by FDA under an EUA. This product has been authorized only for the home collection and maintenance of anterior nasal swab specimens as an aid in detection of nucleic acid from SARS-CoV-2, not for any other viruses or pathogens. The emergency use of this product is only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of medical devices under Section 564(b)(1) of the Federal Food, Drug, and Cosmetic Act, 21 U.S.C. § 360bbb-3(b)(1), unless the declaration is terminated or authorization is revoked sooner.

For Use by individuals 18 years of age and older when self-collected  
For Use by individuals 16 years of age and older when collected under adult supervision  
For Use by individuals 2 years of age and older when collected with adult assistance

Check the order status any time by going to [everlywell.com/orders](https://everlywell.com/orders)

1. Once the sample arrives at the lab, it will pass a quality check before processing begins. We'll contact you if there are any issues.
2. A board-certified physician will review the test taker's results and securely transmit them to us.
3. We'll email you as soon as the results arrive.

## What's next?

## COVID-19 Test Home Collection Kit DTC

### Instructions

The kit includes:



Swab and tube




Kit ID stickers



Biohazard bag with Absorbent Sheet



Return box

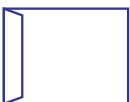

Return envelope






Return label



Alcohol prep pad

For Use Under Emergency Use Authorization. For In Vitro Diagnostic Use.

## Questions?

 +1 855-463-1670  
 [covid19@everlywell.com](mailto:covid19@everlywell.com)  
 [support.everlywell.com](https://support.everlywell.com)


# Important Reminder

Be sure to complete these 3 steps before mailing the kit.

## Register kit



## Fill out kit ID sticker

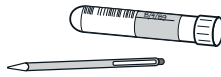

## Ship on day of collection



Visit [www.everlywell.com/register](https://www.everlywell.com/register) and complete registration.

The lab **can only** process the sample if you:

- ☐ Register the test taker's sample at [everlywell.com/register](https://www.everlywell.com/register)
- ☐ Fill out ID sticker with test taker's information and affix to the sample tube
- ☐ Ship sample the same day of collection (Monday-Friday)

## Sample Collection Instructions

Please read all instructions fully prior to collecting the specimen. Follow all instructions closely to ensure the collected specimen is acceptable for processing. Failure to follow all instructions may lead to the specimen being rejected or to incorrect results. Make sure you can ship the same day the sample is collected (Monday-Friday).

1



### Register

Register the kit at [everlywell.com/register](https://www.everlywell.com/register) using the test taker's information before collecting the sample. Do not register until you are ready to collect the sample because you will be asked to enter the time of collection. **Collect Monday-Friday as samples cannot be shipped on the weekend.** The lab cannot process the sample if this step is missed.

2



### Watch video

For best results, watch our collection video at [everlywell.com/collect](https://www.everlywell.com/collect) before you start. **If you are under 18, please ask your parent or guardian for assistance with the packaging and return shipment of your specimen.**


3



### Locate UPS drop box

Visit the UPS website to view drop box locations and pick up schedules. **Your sample must be dropped off at the drop box before the last pickup on the day it is collected. Only collect samples on Monday-Friday as samples cannot be shipped on the weekend.**


4

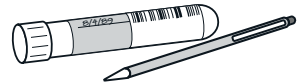

### Fill out kit ID sticker

Using a ballpoint pen or fine tip permanent marker, **write the test taker's full name, date of birth, and collection date on the ID sticker** before placing it on the collection tube. **Do not cover the barcode sticker** already on the tube. The information must match what you used to register the kit. **Do not pour the liquid out of the tube.**

5

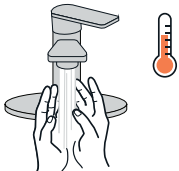

### Prep space and wash hands

Disinfect surfaces where the specimen will be collected. **The test taker should blow their nose.** The person collecting the sample should thoroughly wash their hands with soap and water for 20 seconds prior to collection.


6



### Prepare to collect

Before inserting the swab, locate the perforation on the swab (near the tip), as this will be needed after the sample is collected. The test taker should tilt their head back slightly.


7

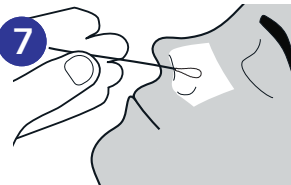

### Insert nasal swab

Hold the swab below the perforation line. **Insert the soft tip of the swab into the nostril. That's usually 1/2 to 3/4 of an inch for an adult and less than 1/2 an inch for younger children. DO NOT go beyond this depth. If you are collecting a specimen from a child, be careful not to insert the swab too far into the child's nostril.**

8

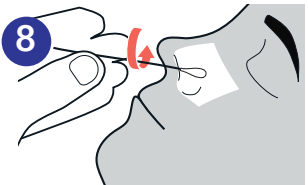

### Collect swab sample

Rotate the swab 4 times all the way around the nasal wall, applying slight pressure and taking approximately 15 seconds to collect the sample. Remove swab. **Repeat this process in the other nostril using the same swab.**

9



### Place the swab sample in the tube

Place the swab sample (tip down) in the tube without spilling the liquid. Break the swab at the perforated line (near the tip) and leave in tube. **Tightly seal cap.** Liquid volume may not completely cover the swab tip.

10



### Place tube into bag

Place tube into protective biohazard bag with the absorbent sheet. **Follow return instructions on the reverse.**