



JVCKENWOOD Corporation
% Hideki Tengeiji
Senior Manager
3-12, Moriya-cho, Kanagawa-ku
Yokohama-shi, Kanagawa 221-0022
JAPAN

December 16, 2021

Re: K212985

Trade/Device Name: 6MP Color LCD Monitor CL-S600
Regulation Number: 21 CFR 892.2050
Regulation Name: Medical Image Management And Processing System
Regulatory Class: Class II
Product Code: PGY
Dated: September 27, 2021
Received: September 30, 2021

Dear Hideki Tengeiji:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. Although this letter refers to your product as a device, please be aware that some cleared products may instead be combination products. The 510(k) Premarket Notification Database located at <https://www.accessdata.fda.gov/scripts/cdrh/cfdocs/cfpmn/pmn.cfm> identifies combination product submissions. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803) for

devices or postmarketing safety reporting (21 CFR 4, Subpart B) for combination products (see <https://www.fda.gov/combination-products/guidance-regulatory-information/postmarketing-safety-reporting-combination-products>); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820) for devices or current good manufacturing practices (21 CFR 4, Subpart A) for combination products; and, if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to <https://www.fda.gov/medical-devices/medical-device-safety/medical-device-reporting-mdr-how-report-medical-device-problems>.

For comprehensive regulatory information about medical devices and radiation-emitting products, including information about labeling regulations, please see Device Advice (<https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance>) and CDRH Learn (<https://www.fda.gov/training-and-continuing-education/cdrh-learn>). Additionally, you may contact the Division of Industry and Consumer Education (DICE) to ask a question about a specific regulatory topic. See the DICE website (<https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance/contact-us-division-industry-and-consumer-education-dice>) for more information or contact DICE by email (DICE@fda.hhs.gov) or phone (1-800-638-2041 or 301-796-7100).

Sincerely,

For

Thalia T. Mills, Ph.D.
Director
Division of Radiological Health
OHT7: Office of In Vitro Diagnostics
and Radiological Health
Office of Product Evaluation and Quality
Center for Devices and Radiological Health

Enclosure

Indications for Use

510(k) Number (if known)
K212985

Device Name
6MP Color LCD Monitor CL-S600

Indications for Use (Describe)

CL-S600 is intended to be used in displaying and viewing medical images for diagnosis by trained medical practitioners or certified personnel. It is not meant to be used in digital mammography.

Type of Use (Select one or both, as applicable)

Prescription Use (Part 21 CFR 801 Subpart D)

Over-The-Counter Use (21 CFR 801 Subpart C)

CONTINUE ON A SEPARATE PAGE IF NEEDED.

This section applies only to requirements of the Paperwork Reduction Act of 1995.

DO NOT SEND YOUR COMPLETED FORM TO THE PRA STAFF EMAIL ADDRESS BELOW.

The burden time for this collection of information is estimated to average 79 hours per response, including the time to review instructions, search existing data sources, gather and maintain the data needed and complete and review the collection of information. Send comments regarding this burden estimate or any other aspect of this information collection, including suggestions for reducing this burden, to:

Department of Health and Human Services
Food and Drug Administration
Office of Chief Information Officer
Paperwork Reduction Act (PRA) Staff
PRASStaff@fda.hhs.gov

"An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB number."

510(k) SUMMARY

K212985

Submitted Information: JVCKENWOOD Corporation
3-12, Moriya-cho, Kanagawa-ku,
Yokohama-shi, Kanagawa, 221-0022 Japan

Contact Person: Hideki Tengeiji, Senior Manager
Email: tengeiji.hideki@jvckenwood.com
Tel: +81-45-450-2715
Fax: +81-45-450-1926

Date Prepared: September 14, 2021

Device Name: 6MP Color LCD Monitor CL-S600

Common Name: display, diagnostic radiology

Classification Name: Class II
(Part 892 Radiology Devices
Sec. 892.2050
Medical Image Management and Processing System

Predicate Device: 30 inch (76.8cm) Color LCD Monitor CCL650i2
(CL30650/ K140214)

Device Description: 30 inch Color LCD Monitor
3280 x 2080 (landscape)

- High-luminance color LCD panel, which has wide view angle, is used for this product. It is designed for medical image display.
- Luminance stabilization function composed with luminance sensor and luminance control circuit always observes the luminance and makes it stable.
- Images are faithfully displayed along grayscale characteristics (DICOM GSDF) based on the calibrated data stored to the lookup table of the monitor.
- Luminance and the color mura correction functions will help achieve uniformity on the whole screen.

JVCKENWOOD Corporation

Healthcare Business Division
3-12, Moriya-cho, Kanagawa-ku,
Yokohama-shi, Kanagawa, 221-0022 Japan

JVCKENWOOD

Cybersecurity: FDA guidance located at <https://www.fda.gov/media/86174/download>, is followed for cybersecurity concerns.

Intended Use: CL-S600 is intended to be used in displaying and viewing medical images for diagnosis by trained medical practitioners or certified personnel. It is not meant to be used in digital mammography.

Substantial Equivalence: CL-S600 shares the same technical characteristics, application, and intended use as our predicate device CCL650i2 (CL30650/ K140214)

JVCKENWOOD Corporation
Healthcare Business Division
3-12, Moriya-cho, Kanagawa-ku,
Yokohama-shi, Kanagawa, 221-0022 Japan

Device Description & Substantial Equivalence Comparison

	Predicate device LCD Monitor CL30650	Proposed device LCD Monitor CL-S600	Explanation of Differences
510(k) Number	K140214	—	—
Indication for use	CL30650 is intended to be used in displaying and viewing medical images for diagnosis by trained Medical practitioners. It is not meant to be used in digital mammography.	CL-S600 is intended to be used in displaying and viewing medical images for diagnosis by trained medical practitioners or certified personnel. It is not meant to be used in digital mammography.	—
Display Technology	IPS LCD panel with TFT active-matrix array with LED backlight	IPS LCD panel with TFT active-matrix array with LED backlight	—
Screen size	Diagonal: 30" Aspect ratio: 16:9.99	Diagonal: 30" Aspect ratio: 16:10.15	The display area of the proposed device is wider than the predicate device. Compared to the predicate device, the proposed device has a larger display area.
Backlight type	LED	LED	—
Frame rate and refresh rate	60Hz	60Hz	—
Resolution / Pixel array	6MP (3280 x 2048)	6MP (3280 x 2080)	The resolution of the proposed device is larger than that of the predicate device.
Pixel Pitch	Horizontal: 0.197mm Vertical: 0.197mm	Horizontal: 0.197mm Vertical: 0.197mm	—
Subpixel pattern	Stripe RGB	Stripe RGB	—
Pixel aperture ratio	43.0%	46.5 %	Almost the same.
Subpixel driving (spatial and temporal dithering)	N/A	N/A	—
Display Interface	Input: DVI-D x2 DisplayPort x2 Output: N/A	Input: DisplayPort x2 Output: DisplayPort x1	The proposed device is not equipped with a DVI-D input, but this is not a problem since graphics cards are no longer equipped with a DIV-D output.
Video bandwidth	Dot clock: 443.52 MHz	Dot clock: 443.52 MHz	—

	Predicate device LCD Monitor CL30650	Proposed device LCD Monitor CL-S600	Explanation of Differences
User controls	Input signal switch EDID switch Dynamic gamma AUTO TEXT Test pattern Configuration switch, INPUT SOURCE AUTO SCANNING	Input signal switch Dynamic gamma AUTO TEXT Test pattern Configuration switch Pixel Enhancer Turbo Luminance Dynamic Range Extension Auto Config Select Human Presence Sensor Reading Light USB Power	The functions in the predicate device are also available in the proposed device. Regarding "EDID switch", the proposed device can automatically switch EDID.
Ambient light sensing	Built-in Sensor (For correction during calibration)	Built-in Sensor (For correction during calibration)	—
Touch-screen technology	N/A	N/A	—
Luminance calibration tools / Quality-control procedures	Hardware: Integrated sensor External sensor Software: QA Medivisor / Medivisor NX FCAL	Hardware: Integrated sensor External sensor Software: QA Medivisor Agent FCAL	—
Additional Software/Firmware	N/A	N/A	—

Physical Laboratory Tests

Performance test items in the guidance	Test method(s)
a. Spatial resolution	The bar pattern is displayed and captured by a digital camera equipped with a macro lens. The MTF is calculated with the captured data.
b. Pixel defects (maximum counts, allowed defect types, and locations)	ISO 13406-2 IDMS 1.03, 7.6 DEFECTIVE PIXELS Pixel defects are counted based on the ISO13406-2, 3.4.13 table 3.
c. Artifacts	AAPM-TG18, 4.9 Miscellaneous Tests IDMS 1.03, 4.6 ARTIFACTS & IRREGULARITIES
d. Temporal response	IDMS 1.03, 10.2.3 GRAY-TO-GRAY RESPONSE TIME Rise and fall time constants at four grayscale intervals (0-100%, 5-95%, 10-90%, 40-60%) are provided by the panel manufacturer.
e. Luminance (maximum, minimum, achievable, and recommended)	L_{min} and L_{max} on the calibrated luminance are confirmed.
f. Conformance to a grayscale-to-luminance function (e.g., DICOM GSDF)	AAPM-TG18, 4.3.5 Advanced Luminance Response Luminance response for 256 levels are measured.
(For mammography displays) g. Luminance at 30° and 45° in diagonal, horizontal, and vertical directions at center and four corners	N/A
(For mammography displays) h. Luminance uniformity or Mura test	N/A
(For mammography displays) i. Stability of luminance and chromaticity response with temperature and time of operation or on-time	N/A
(For mammography displays) j. Spatial noise	N/A
(For mammography displays) k. Reflection coefficient	N/A
(For mammography displays) l. Veiling glare or small-spot contrast	N/A
(For color displays) m. Color tracking (primary colors and color gamut)	Color scale: IDMS 1.03, 6. Gray- and Color-Scale Measurement IDMS 1.03, 5.4 Color-Signal White Color gamut volume: IDMS 1.03, 5.31 Volume-Color-Reproduction Capability
(For color displays) n. Gray tracking (gray shades and white point)	AAPM-TG196 Gray Tracking IEC 62563-1: 2009 + AMD1: 2016 CSV, 7.4.9 Greyscale chromaticity evaluation

Conclusion

As shown above, the intended use of the subject and predicate devices are identical, the technical characteristics are similar, and any differences between the characteristics do not affect the device safety or effectiveness. The results of the performance testing and the verification and validation demonstrate that the subject device is substantially equivalent to the currently marketed predicate device.