



February 15, 2023

Canon, USA Inc.
% Saori Sawaki
Business Manager/Regulatory Consultant
Ken Block Consulting LLC
800 E Campbell Road, Suite 202
RICHARDSON TX 75081

Re: K230175
Trade/Device Name: Digital Radiography CXDI-CS01
Regulation Number: 21 CFR 892.1680
Regulation Name: Stationary X-Ray System
Regulatory Class: Class II
Product Code: MQB
Dated: January 20, 2023
Received: January 20, 2023

Dear Saori Sawaki:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. Although this letter refers to your product as a device, please be aware that some cleared products may instead be combination products. The 510(k) Premarket Notification Database located at <https://www.accessdata.fda.gov/scripts/cdrh/cfdocs/cfpmn/pmn.cfm> identifies combination product submissions. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803) for

devices or postmarketing safety reporting (21 CFR 4, Subpart B) for combination products (see <https://www.fda.gov/combination-products/guidance-regulatory-information/postmarketing-safety-reporting-combination-products>); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820) for devices or current good manufacturing practices (21 CFR 4, Subpart A) for combination products; and, if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to <https://www.fda.gov/medical-devices/medical-device-safety/medical-device-reporting-mdr-how-report-medical-device-problems>.

For comprehensive regulatory information about medical devices and radiation-emitting products, including information about labeling regulations, please see Device Advice (<https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance>) and CDRH Learn (<https://www.fda.gov/training-and-continuing-education/cdrh-learn>). Additionally, you may contact the Division of Industry and Consumer Education (DICE) to ask a question about a specific regulatory topic. See the DICE website (<https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance/contact-us-division-industry-and-consumer-education-dice>) for more information or contact DICE by email (DICE@fda.hhs.gov) or phone (1-800-638-2041 or 301-796-7100).

Sincerely,

A large, light blue watermark of the letters "FDA" is visible in the background. Overlaid on this is a handwritten signature in black ink that reads "Lu Jiang".

Lu Jiang, Ph.D.
Assistant Director
Diagnostic X-Ray Systems Team
DHT8B: Division of Radiological Imaging Devices
and Electronic Products
OHT8: Office of Radiological Health
Office of Product Evaluation and Quality
Center for Devices and Radiological Health

Enclosure

Indications for Use

510(k) Number (if known)
K230175

Device Name
DIGITAL RADIOGRAPHY CXDI-CS01

Indications for Use (Describe)

The DIGITAL RADIOGRAPHY CXDI-CS01 provides digital image capture for conventional film/screen radiographic examinations. This device is intended to capture, for display, radiographic images of human anatomy, and to replace radiographic film/screen systems in all general purpose diagnostic procedures. This device is not intended for mammography applications.

Type of Use (Select one or both, as applicable)

Prescription Use (Part 21 CFR 801 Subpart D)

Over-The-Counter Use (21 CFR 801 Subpart C)

CONTINUE ON A SEPARATE PAGE IF NEEDED.

This section applies only to requirements of the Paperwork Reduction Act of 1995.

DO NOT SEND YOUR COMPLETED FORM TO THE PRA STAFF EMAIL ADDRESS BELOW.

The burden time for this collection of information is estimated to average 79 hours per response, including the time to review instructions, search existing data sources, gather and maintain the data needed and complete and review the collection of information. Send comments regarding this burden estimate or any other aspect of this information collection, including suggestions for reducing this burden, to:

Department of Health and Human Services
Food and Drug Administration
Office of Chief Information Officer
Paperwork Reduction Act (PRA) Staff
PRASStaff@fda.hhs.gov

"An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB number."



510(k) Summary

K230175

Applicant/ Sponsor: Canon Inc.
9-1 Imaikami-cho
Nakahara-ku, Kanagawa 211-8501 JAPAN

Contact Person: Mr. Akira Hirai
General Manager
TEL: 81-3-3758-2111
FAX: 81-44-739-6695
hirai.akira@mail.canon

Date Prepared: February 10, 2023

Subject Device Manufacturer: Canon Inc.
Trade Name: DIGITAL RADIOGRAPHY CXDI-CS01
Common Name: Solid State X-Ray Imager (Flat Panel/Digital Imager)
Classification Name: Stationary X-ray System
Product Code: MQB
Regulation: 892.1680, Stationary x-ray system
Class: II

Predicate Device: Clearance: K222661 dated September 27, 2022
Manufacturer: Canon Inc.
Trade Name: DIGITAL RADIOGRAPHY CXDI-CS01
Common Name: Solid State X-Ray Imager (Flat Panel/Digital Imager)
Classification Name: Stationary X-ray System
Product Code: MQB
Regulation: 892.1680, Stationary x-ray system
Class: II

Device Description: The DIGITAL RADIOGRAPHY CXDI-CS01 (hereinafter referred to as CXDI-CS01) is a solid-state x-ray imager. The CXDI-CS01 is a series of detectors and, in the predicate device submission (K222661), consists of the CXDI-702C Wireless, CXDI-402C Wireless, CXDI-710C Wireless, CXDI-810C Wireless, CXDI-410C Wireless, DIGITAL RADIOGRAPHY CXDI-Pro (CXDI-703C Wireless and CXDI-403C Wireless), and DIGITAL RADIOGRAPHY CXDI-Elite (CXDI-720C Wireless).
The CXDI-CS01 intercepts x-ray photons, and the scintillator emits visible spectrum photons that illuminate an array of photodetectors that create electrical signals. After the electrical signals are generated, the signals are converted to digital values. The digital values are sent to the PC via a wired or wireless connection, converted to images with the CXDI Control Software, and then displayed on the PC/monitors. The PC/monitors used with the CXDI-CS01 are not a part of this submission.

The subject of this Special 510(k) submission is the changes to the predicate CXDI-CS01, which include the addition of the DIGITAL RADIOGRAPHY CXDI-Pro detector (CXDI-803C Wireless) and the DIGITAL RADIOGRAPHY CXDI-Elite detectors (CXDI-820C Wireless and CXDI-420C Wireless) to the CXDI-CS01 series of detectors, a firmware update of the DIGITAL RADIOGRAPHY CXDI-Pro and DIGITAL RADIOGRAPHY CXDI-Elite detectors from 01.01.03.00 to 01.02.00.01 and from F2.00.03.00 to 02.01.04.00 respectively, and a CXDI Control



510(k) Summary

K230175

Software version update from V3.10.2.2 to V3.10.2.6 to fix bugs. Together, these changes make up the CXDI-CS01.

Indications for Use:

The DIGITAL RADIOGRAPHY CXDI-CS01 provides digital image capture for conventional film/screen radiographic examinations. This device is intended to capture, for display, radiographic images of human anatomy, and to replace radiographic film/screen systems in all general purpose diagnostic procedures. This device is not intended for mammography applications.

Summary of Technological Characteristics:

Comparison with the predicate device shows the technological characteristics of the proposed CXDI-CS01 is substantially equivalent to the predicate device. The major differences between the proposed CXDI-CS01 and the predicate are differences in the components and a newer version of the CXDI control software. The proposed indications for use statement is identical to the indications for use statement of the predicate device.

	New Device	Predicate Device
Trade Name	DIGITAL RADIOGRAPHY CXDI-CS01	DIGITAL RADIOGRAPHY CXDI-CS01
510(k) Submitter [Number]	Canon Inc. [K230175]	Canon Inc. [K222661]
Indications for Use	The DIGITAL RADIOGRAPHY CXDI-CS01 provides digital image capture for conventional film/screen radiographic examinations. This device is intended to capture, for display, radiographic images of human anatomy, and to replace radiographic film/screen systems in all general purpose diagnostic procedures. This device is not intended for mammography applications.	The DIGITAL RADIOGRAPHY CXDI-CS01 provides digital image capture for conventional film/screen radiographic examinations. This device is intended to capture, for display, radiographic images of human anatomy, and to replace radiographic film/screen systems in all general purpose diagnostic procedures. This device is not intended for mammography applications.
Application	General Radiography	General Radiography
Software	CXDI Control Software V3.10.2.6	CXDI Control Software V3.10.2.2
Components	Detectors Battery Pack (LB-4A) Battery Charger (BC-1A/BC-01) Multi Box (MB-4A/MB-02) Power Box (PB-01) Ready Indicator (RI-3A) Status Indicator (SI-4A/SI-01) Wiring Cable (WC-4A/WC-01) PC Connection Cable (CP-4A/CP-01) Docking Station (DS-4A) Detector Stand (DS-01)	Detectors Battery Pack (LB-4A) Battery Charger (BC-1A/BC-01) Multi Box (MB-4A/MB-02) Power Box (PB-01) Ready Indicator (RI-3A) Status Indicator (SI-4A/SI-01) Wiring Cable (WC-4A/WC-01) PC Connection Cable (CP-4A/CP-01) Docking Station (DS-4A) Detector Stand (DS-01)
Detector	CXDI-702C Wireless CXDI-402C Wireless CXDI-710C Wireless CXDI-810C Wireless CXDI-410C Wireless CXDI-Pro: CXDI-703C Wireless CXDI-803C Wireless CXDI-403C Wireless CXDI-Elite: CXDI-720C Wireless CXDI-420C Wireless CXDI-820C Wireless	CXDI-702C Wireless CXDI-402C Wireless CXDI-710C Wireless CXDI-810C Wireless CXDI-410C Wireless CXDI-Pro: CXDI-703C Wireless CXDI-403C Wireless CXDI-Elite: CXDI-720C Wireless

	New Device	Predicate Device
Trade Name	DIGITAL RADIOGRAPHY CXDI-CS01	DIGITAL RADIOGRAPHY CXDI-CS01
Scintillator	CsI(Tl) [Cesium Iodide doped with Thallium]	CsI(Tl) [Cesium Iodide doped with Thallium]
Pixel Pitch	<u>CXDI-702C Wireless, CXDI-402C Wireless, CXDI-710C Wireless, CXDI-810C Wireless, CXDI-410C Wireless, and CXDI-Elite:</u> 125µm <u>CXDI-Pro:</u> 140 µm	<u>CXDI-702C Wireless, CXDI-402C Wireless, CXDI-710C Wireless, CXDI-810C Wireless, CXDI-410C Wireless, and CXDI-Elite:</u> 125µm <u>CXDI-Pro:</u> 140 µm
Spatial Resolution	<u>CXDI-702C Wireless, CXDI-402C Wireless, CXDI-710C Wireless, CXDI-810C Wireless, CXDI-410C Wireless, and CXDI-Pro:</u> 35% [MTF@2lp/mm] <u>CXDI-Elite:</u> 45% [MTF@2lp/mm]	<u>CXDI-702C Wireless, CXDI-402C Wireless, CXDI-710C Wireless, CXDI-810C Wireless, CXDI-410C Wireless, and CXDI-Pro:</u> 35% [MTF@2lp/mm] <u>CXDI-Elite:</u> 45% [MTF@2lp/mm]
DQE	<u>CXDI-702C Wireless:</u> 58% [@0.5 lp/mm, 1 mR] <u>CXDI-402C Wireless:</u> 58% [@0.5 lp/mm, 3.5 uGy] <u>CXDI-710C Wireless, CXDI-810C Wireless, and CXDI-410C Wireless:</u> 60% [@0 lp/mm, 4 uGy] <u>CXDI-Pro:</u> 58% [@0.5 lp/mm, 3.5 uGy] <u>CXDI-Elite:</u> 67% [@0.5 lp/mm, 3.5 uGy]	<u>CXDI-702C Wireless:</u> 58% [@0.5 lp/mm, 1 mR] <u>CXDI-402C Wireless:</u> 58% [@0.5 lp/mm, 3.5 uGy] <u>CXDI-710C Wireless, CXDI-810C Wireless, and CXDI-410C Wireless:</u> 60% [@0 lp/mm, 4 uGy] <u>CXDI-Pro:</u> 58% [@0.5 lp/mm, 3.5 uGy] <u>CXDI-Elite:</u> 67% [@0.5 lp/mm, 3.5 uGy]
IP Level	<u>CXDI-702C Wireless and CXDI-402C Wireless IP54</u> <u>CXDI-710C Wireless, CXDI-810C Wireless, and CXDI-410C Wireless:</u> IPX7 <u>CXDI-Pro:</u> IP55 <u>CXDI-Elite:</u> IP57	<u>CXDI-702C Wireless and CXDI-402C Wireless IP54</u> <u>CXDI-710C Wireless, CXDI-810C Wireless, and CXDI-410C Wireless</u> IPX7 <u>CXDI-Pro:</u> IP55 <u>CXDI-Elite:</u> IP57
Wireless Communication	<u>CXDI-702C Wireless, CXDI-402C Wireless, CXDI-710C Wireless, CXDI-810C Wireless, and CXDI-410C Wireless:</u> IEEE 802.11n 2.4GHz/5GHz <u>CXDI-Pro and CXDI-Elite:</u> IEEE 802.11a/b/g/n/ac 2.4GHz/5GHz Bluetooth Low Energy	<u>CXDI-702C Wireless, CXDI-402C Wireless, CXDI-710C Wireless, CXDI-810C Wireless, and CXDI-410C Wireless:</u> IEEE 802.11n 2.4GHz/5GHz <u>CXDI-Pro and CXDI-Elite:</u> IEEE 802.11a/b/g/n/ac 2.4GHz/5GHz Bluetooth Low Energy
Photographing Mode	<u>CXDI-702C Wireless, CXDI-402C Wireless, and CXDI-Pro:</u> Standard Synchronization Mode and Non Generator Connection Mode <u>CXDI-710C Wireless, CXDI-810C Wireless, and CXDI-410C Wireless:</u> Standard Synchronization Mode, Non Generator Connection Mode, and Standalone Mode <u>CXDI-Elite:</u> Standard Synchronization Mode, Standard Synchronization Mode with Built in AEC Assistance, Non Generator Connection Mode, and Standalone Mode	<u>CXDI-702C Wireless, CXDI-402C Wireless, and CXDI-Pro:</u> Standard Synchronization Mode and Non Generator Connection Mode <u>CXDI-710C Wireless, CXDI-810C Wireless, and CXDI-410C Wireless:</u> Standard Synchronization Mode, Non Generator Connection Mode, and Standalone Mode <u>CXDI-Elite:</u> Standard Synchronization Mode, Standard Synchronization Mode with Built in AEC Assistance, Non Generator Connection Mode, and Standalone Mode

510(k) Summary**K230175**

Summary of
Non-Clinical/
Test Data:

The fundamental scientific technology of the CXDI-CS01 has not been modified. The major change is the addition of new detectors and component.

Evaluation of the addition of the new detectors confirmed that the change did not impact the conformance of CXDI-CS01 with the U.S. Performance Standard for radiographic equipment and relevant voluntary safety standards for Electrical safety and Electromagnetic Compatibility testing, specifically IEC standards 60601-1, 60601-1-2, 60601-1-3, 60601-1-6, 60601-2-54, 62220-1, 62304, and 62366. The design control activities and biocompatibility evaluation successfully demonstrated that the modifications to the predicate device raise no new questions regarding either safety or effectiveness. Clinical testing is not necessary for the current submission, based on the minor differences from the predicate device. Adequate detector bench testing should be sufficient to demonstrate that the subject detector, CXDI-CS01, works as intended. Therefore, the verification/validation activities conducted support a determination of substantial equivalence for the CXDI-CS01.

The proposed device follows the applicable elements of the following FDA guidance documents: *Guidance for the Submission of 510(k)s for Solid State X-ray Imaging Devices*, *Guidance for the Content of Premarket Submissions for Software contained in Medical Devices*, *Content of Premarket Submissions for Management of Cybersecurity in Medical Devices*, *Use of International Standard ISO 10993-1, "Biological evaluation of medical devices - Part 1: Evaluation and testing within a risk management process"*, *Radio Frequency Wireless Technology in Medical Devices*, and *Pediatric Information for X-ray Imaging Device Premarket Notifications*.

Conclusion:

Canon Inc. considers the DIGITAL RADIOGRAPHY CXDI-CS01 device to be substantially equivalent to the predicate device listed above. This conclusion is based on the similarities in primary intended use, principles of operation, functional design, and established medical use.