

Instructions for Use:

1. Facing the respirator on the side of a silicone seal, and make the nose clip in the top of the respirator, then hold the respirator with hands to make it fixed with the face and against the chin.
2. Pull the elastic bands over the ear.
3. Adjust the respirator position so that the silicone seal around the respirator is comfortable with the face.
4. Adjust the elastic band and the buckle to the appropriate felt strength of elastic band.
5. Check the fit of the respirator to the face:
 - a) Cover the respirator with both hands to avoid affecting the position of the respirator on the face.
 - b) Make a quick inspiration. If air leaks from the edge of the respirator, please re-adjust the elastic band; if you can not get a good fit, repeat steps 1-4.

