

Instructions for Use:

1. Hold the mask by the ear loops. Place a loop around each ear, making sure the interior of the face mask is facing the user.
2. Make sure the nose bridge is to the top of the mask.
3. Mold or pinch the nose bridge to the shape of the user's nose.
4. Adjust the respirator position so that the silicone seal around the respirator is comfortably resting on the user's face.
5. Adjust the elastic band and the buckle to the appropriate felt strength of elastic band.
6. Pull the bottom of the mask to cover the user's mouth and chin, while making sure the user's nose is covered.
7. Check the fit of the respirator to the face:
 - a. Cover the respirator with both hands to avoid affecting the position of the respirator on the face.
 - b. Breathe in and out and if air leaks from the edge of the respirator, please re-adjust the elastic band; if the user cannot get a good fit, repeat steps 1-6.

