SDNA - 1000

2mL Saliva Collection Device - Instructions For Use (IFU)

For unsupervised specimen self-collection by a layperson 18 years and older or for specimen collection by a healthcare worker from individuals of any age.

Do NOT eat, drink, smoke, or chew gum for 30 minutes before giving your saliva sample.

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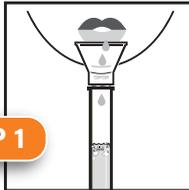
STEP 1

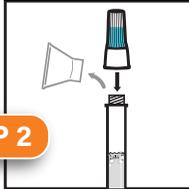
Fill the tube with saliva to the black wavy line. Fill the tube until your saliva (not including bubbles) is at or just above the wavy line. **DO NOT OVERFILL.**

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STEP 2

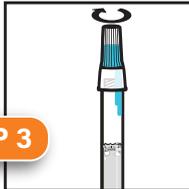
Replace the funnel with the fluid cap. Remove the funnel from the tube. Screw on the enclosed cap **TIGHTLY** to release the solution that will stabilize the DNA in your saliva.

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STEP 3

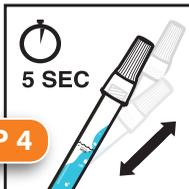
Firmly screw cap down to release solution and seal tube. You will know it works when the blue solution from the cap is released into the tube. Firmly tighten cap to assure the cap and tube is completely sealed.

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STEP 4

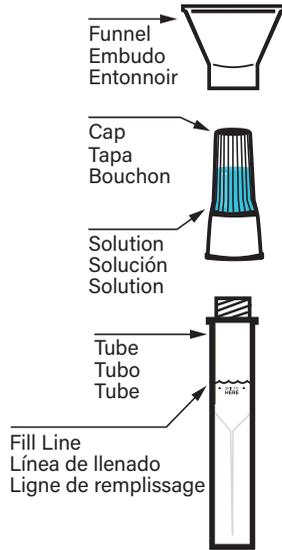
Shake the tube for at least five seconds. This will ensure your sample mixes thoroughly with the stabilizing solution.

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SDNA-1000 SALIVA COLLECTION DEVICE - Safety Information

- Children under 12 should be supervised
- Do not clean with any substance containing bleach
- Do not use if kit is expired
- Cap is a choking hazard. Keep out of reach of children. Wash with water if liquid solution comes in contact with eyes or skin. **DO NOT INGEST LIQUID.**
- Avoid contact with skin and eyes. If contact with body occurs, rinse with water. If irritation persists, seek medical advice.
- This kit is designed for the collection of human saliva samples.
- This product has not been FDA cleared or approved, but, has been authorized by FDA under an EUA;

This product has been authorized only for the collection and maintenance of saliva as an aid in detection of nucleic acid from SARS-CoV-2, not for any other viruses or pathogens;

The emergency use of this product is only authorized for the duration of the declaration that circumstances exist justifying the authorization of emergency use of medical devices during the COVID-19 outbreak under Section 564(b)(1) of the Federal Food, Drug, and Cosmetic Act, 21 U.S.C. § 360bbb-3(b)(1), unless the declaration is terminated or authorization is revoked sooner.

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Spectrum Solutions
12248 S Lone Peak Parkway, #106
Draper, UT 84020
spectrum**solution**.com
Storage 15°C ± 30°C

(REF) 2019162

(CE) (MD) Quality First International OÜ
Laki 30
12915 Tallinn
Estonia
Tel.: +372 61041 96

Label Legend:

- ☞ Consult instructions for use
- 📅 Use by date (Collect saliva by)
- IVD In vitro diagnostic medical device
- 1001 Catalog number
- ⚠ Caution, consult instructions for use
- 🌡 Temperature limit
- ☞ (REF) Authorized representative
- 🏢 Manufacturer
- 1001 Batch code
- ☞ Prescription Only

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CUSTOMER: Spectrum Solutions	Black	
CREATED DATE: 3 19 2021	PROOF #: 1	PMS 151
SIZE: 0	DESIGNER: LTB	PMS 7621
APPROVED BY:	DATE:	PMS 3115
FILE: SDNA 1000 2-Sided IFU Insert		PMS 447
	DOC#	PS-2019077
		v. 5.3