

Instructions for Use:

1. Facing the respirator on the side of the silicone nose bridge, making sure the nose bridge is on top of the respirator, then hold the respirator with one hand to make it fixed with the face and against the chin.
2. Pull an elastic band over each ear.
3. Adjust the respirator position so that the silicone nose bridge rests on the nose and the respirator is comfortably on the face.
4. Adjust the elastic band and the buckle to the appropriate tension strength of elastic band.
5. Check the fit of the respirator to the face:
 - a) Cover the respirator with both hands to avoid affecting the position of the respirator on the face.
 - b) Make a quick inspiration. If air leaks from the edge of the respirator, please re-adjust the elastic band; if you can not get a good fit, repeat steps 1-4.
 - c) If the mask is damaged or cannot be closed, do not wear the mask into the contaminated area. Seek help or replace the mask in time.

