



August 19, 2022

W&H Sterilization s.r.l.
% Angela Blackwell
Senior Consultant
Blackwell Device Consulting
P.O. Box 718
Gresham, Oregon 97030-0172

Re: K213758

Trade/Device Name: Lexa PLUS RIS-311
Regulation Number: 21 CFR 880.6880
Regulation Name: Steam Sterilizer
Regulatory Class: Class II
Product Code: FLE
Dated: August 16, 2022
Received: August 17, 2022

Dear Angela Blackwell:

We have reviewed your Section 510(k) premarket notification of intent to market the device referenced above and have determined the device is substantially equivalent (for the indications for use stated in the enclosure) to legally marketed predicate devices marketed in interstate commerce prior to May 28, 1976, the enactment date of the Medical Device Amendments, or to devices that have been reclassified in accordance with the provisions of the Federal Food, Drug, and Cosmetic Act (Act) that do not require approval of a premarket approval application (PMA). You may, therefore, market the device, subject to the general controls provisions of the Act. Although this letter refers to your product as a device, please be aware that some cleared products may instead be combination products. The 510(k) Premarket Notification Database located at <https://www.accessdata.fda.gov/scripts/cdrh/cfdocs/cfpmn/pmn.cfm> identifies combination product submissions. The general controls provisions of the Act include requirements for annual registration, listing of devices, good manufacturing practice, labeling, and prohibitions against misbranding and adulteration. Please note: CDRH does not evaluate information related to contract liability warranties. We remind you, however, that device labeling must be truthful and not misleading.

If your device is classified (see above) into either class II (Special Controls) or class III (PMA), it may be subject to additional controls. Existing major regulations affecting your device can be found in the Code of Federal Regulations, Title 21, Parts 800 to 898. In addition, FDA may publish further announcements concerning your device in the Federal Register.

Please be advised that FDA's issuance of a substantial equivalence determination does not mean that FDA has made a determination that your device complies with other requirements of the Act or any Federal statutes and regulations administered by other Federal agencies. You must comply with all the Act's

requirements, including, but not limited to: registration and listing (21 CFR Part 807); labeling (21 CFR Part 801); medical device reporting (reporting of medical device-related adverse events) (21 CFR 803) for devices or postmarketing safety reporting (21 CFR 4, Subpart B) for combination products (see <https://www.fda.gov/combination-products/guidance-regulatory-information/postmarketing-safety-reporting-combination-products>); good manufacturing practice requirements as set forth in the quality systems (QS) regulation (21 CFR Part 820) for devices or current good manufacturing practices (21 CFR 4, Subpart A) for combination products; and, if applicable, the electronic product radiation control provisions (Sections 531-542 of the Act); 21 CFR 1000-1050.

Also, please note the regulation entitled, "Misbranding by reference to premarket notification" (21 CFR Part 807.97). For questions regarding the reporting of adverse events under the MDR regulation (21 CFR Part 803), please go to <https://www.fda.gov/medical-devices/medical-device-safety/medical-device-reporting-mdr-how-report-medical-device-problems>.

For comprehensive regulatory information about medical devices and radiation-emitting products, including information about labeling regulations, please see Device Advice (<https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance>) and CDRH Learn (<https://www.fda.gov/training-and-continuing-education/cdrh-learn>). Additionally, you may contact the Division of Industry and Consumer Education (DICE) to ask a question about a specific regulatory topic. See the DICE website (<https://www.fda.gov/medical-devices/device-advice-comprehensive-regulatory-assistance/contact-us-division-industry-and-consumer-education-dice>) for more information or contact DICE by email (DICE@fda.hhs.gov) or phone (1-800-638-2041 or 301-796-7100).

Sincerely,

for Clarence W. Murray, III, Ph.D.
Assistant Director
DHT4B: Division of Infection Control
and Plastic Surgery Devices
OHT4: Office of Surgical
and Infection Control Devices
Office of Product Evaluation and Quality
Center for Devices and Radiological Health

Enclosure

Indications for Use

510(k) Number (if known)

K213758

Device Name

Lexa PLUS RIS-311

Indications for Use (Describe)

RIS-311 is a dynamic-air-removal (pre-vacuum) table-top steam sterilizer intended for use by a healthcare provider to sterilize medical products by means of pressurized steam. It is suitable for the sterilization of medical and dental instruments that are validated to be sterilized by steam. The RIS-311 has not been designed to sterilize liquid loads, biomedical waste or materials not compatible with steam sterilization. The processing of such loads may result in incomplete sterilization and/or damage to the autoclave.

Key program features, including sterilization time, temperature and recommended load type are listed in the following table:

Program	Type of Load and Load Weight	Sterilization Temperature	Sterilization Time	Drying Time (Recommended)
Wrapped	Pouched Instruments and Dental Handpieces on trays 14 lbs (6.3 kg) (trays excluded). Instruments and dental handpieces in wrapped cassettes or in trays and wrapped cassettes 16.5 lbs (7.5 kg) (trays and cassettes included). Pouched Textile load on trays 4.4 lbs (2.0 kg) (trays excluded).	270 °F (132 °C)	4 minutes	22 minutes
Low Temperature	Pouched Instruments (solid instruments) on trays 14 lbs (6.3 kg) (trays excluded). Solid instruments in wrapped cassettes or in trays and wrapped cassettes 16.5 lbs (7.5 kg) (trays and cassettes included). Pouched Textile load on trays 4.4 lbs (2.0 kg) (trays excluded).	250 °F (121 °C)	30 minutes	28 minutes
Unwrapped	Instruments and dental handpieces, up to 18 lbs (8.1 kg).	270 °F (132 °C)	3.30 minutes	4 minutes

Type of Use (Select one or both, as applicable)

Prescription Use (Part 21 CFR 801 Subpart D)

Over-The-Counter Use (21 CFR 801 Subpart C)

CONTINUE ON A SEPARATE PAGE IF NEEDED.

This section applies only to requirements of the Paperwork Reduction Act of 1995.

DO NOT SEND YOUR COMPLETED FORM TO THE PRA STAFF EMAIL ADDRESS BELOW.

The burden time for this collection of information is estimated to average 79 hours per response, including the time to review instructions, search existing data sources, gather and maintain the data needed and complete and review the collection of information. Send comments regarding this burden estimate or any other aspect of this information collection, including suggestions for reducing this burden, to:

Department of Health and Human Services
Food and Drug Administration
Office of Chief Information Officer
Paperwork Reduction Act (PRA) Staff
PRAStaff@fda.hhs.gov

“An agency may not conduct or sponsor, and a person is not required to respond to, a collection of information unless it displays a currently valid OMB number.”